CLEARPOINT

NEURO

CLEARPOINT FIDUCIAL

INSTRUCTIONS FOR USE

Table of Contents

- I. **Intended Use**
- II. **Device Description**
- III. **General Warnings and Precautions**
- IV. **Use Instructions**
 - A. **Preparation**
 - B. **Attaching the ClearPoint Fiducial**
 - C. **Storage and Technical Specifications**

I. **Intended Use**

The ClearPoint MR Fiducial Marker (Fiducial) is intended to be used to provide a visual locator on MRI images.

Warning: This device is intended for single use only. Contents of unopened, undamaged package are sterile. Do not re-sterilize.

Caution: Do not use if there is evidence of damage to the fiducial, including leaking or if there is less than two thirds of the fluid remaining in the fiducial.

Caution: Federal (U.S.) law restricts this device to sale by or on the order of a physician.

Note: For a complete procedure description, refer to ClearPoint Neuro ClearPoint User's Guide.

II. **Device Description**

Package Contents:

NGS-BM-05-BR: **ClearPoint MR Fiducial Markers - BR (5)**

III. **General Warnings and Precautions**

The device is intended for single-use-only and is provided sterile. Do not re-sterilize.

General Precautions

- Handle all components using standard hospital sterile practices.

1.5T & 3T Environment Compatibility:

The ClearPoint Fiducial is MR Safe. It contains no metallic components. It is comprised of polymers and fluid.

IV. Use Instructions

A. Preparation


The MR Fiducial Marker is packaged with a double sterile barrier; a sealed Tyvek/mylar pouch is placed inside a larger sealed mylar/Tyvek pouch. Each double pouch contains one MR Fiducial Marker. The MR Fiducial Box contains five (5) of these double pouched MR Fiducial Markers.

Warning: Do not use the MR Fiducial Markers if the packaging is damaged.


Use aseptic transfer if the MR Fiducial Marker is intended to be used in the sterile field.

B. Attaching the MR Fiducial Marker


1. The MR Fiducial Marker comes attached to a non-adhesive white backing, as shown in Figure 1.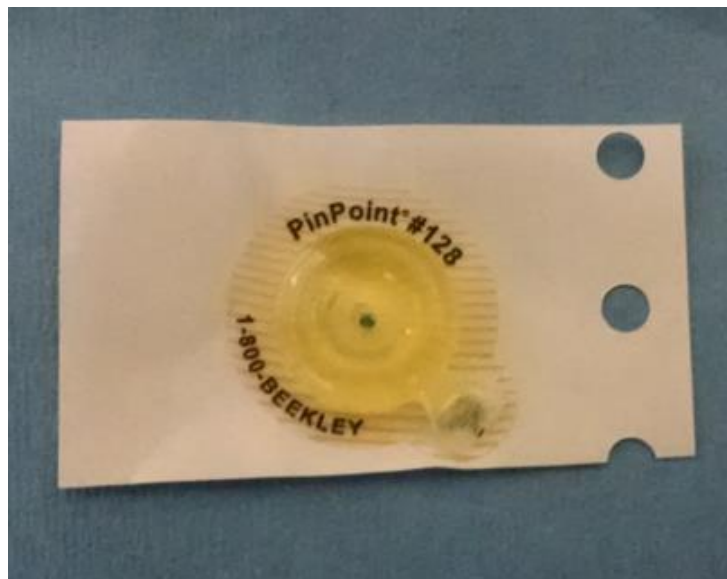


2. Hold the white backing and Fiducial, and fold the backing near the small round tab. This will separate the tab from the backing as shown in Figure 2.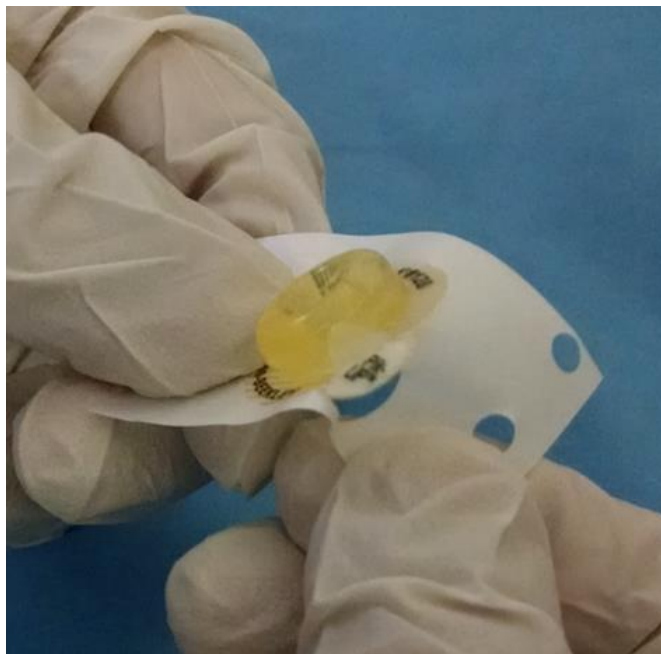

3. While holding the white backing, grab the small round tab and pull it away from the backing. The Fiducial with its adhesive backing will separate from the white backing.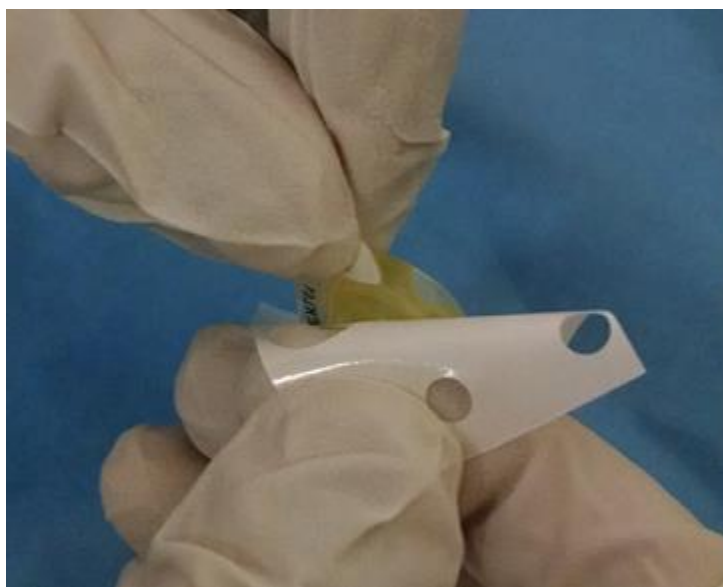

4. Once the fiducial is separated, continue to hold it by the round tab, as that will not stick to gloves.

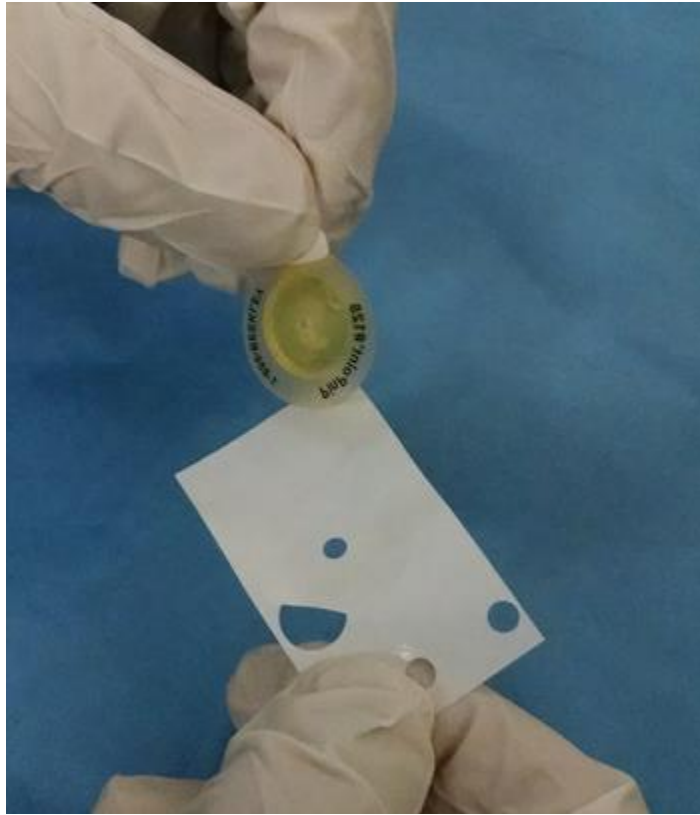5. Place the fiducial on the desired surface, and gently press down to make sure it sticks well.
6. To remove the Fiducial, grasp the round tab and pull the Fiducial off the surface.

C. Storage and Technical Specifications

Storage

- Store in a cool dry place

Technical Specifications

	Outer Diameter (in/mm)	Inner Diameter (in/mm)	Height (in/mm)	Mass (g)
MR Visible portion	0.58 / 14.7	0.056 / 1.4	0.26 / 6.6	N/A
Overall (including Adhesive Backing)	0.88 / 22.3	0.056 / 1.4	0.26 / 6.6	0.88

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ClearPoint Neuro, Inc