# CLEARPOINT<sup>®</sup>

## NEURO

**Multi-Positional Head Fixation Frame and Table Mount, GE 750w**

### **INSTRUCTIONS FOR USE – INSTALLATION**

**NOTE:** These installation instructions are to be used in conjunction with the Universal Multi-Positional Head Fixation Frame Instructions for Use.

**CE**

## Table of Contents

1. Device Description
2. Use Instructions
  - A. Head Fixation Frame Installation

	<b>NOTE:</b> These installation instructions are to be used in conjunction with the Universal Multi-Positional Head Fixation Frame Instructions for Use.
---	--

### 1. Device Description

#### Package Contents:

**CP-MPHFF-332-01: CLEARPOINT<sup>®</sup> Head Fixation Frame (GE 750w - MR Table Compatible)**  
Head Fixation Frame, Upper Fixation Screws (4), Lower Fixation Screws (4),  
Drive Wheels (2)

#### Associated Devices:

**CP-FFSMPU-01: Upper Fixation Frame Screws (replacements in case of damage)**  
Upper Fixation Screws (4)

**CP-FFSMPL-01: Fixation Frame Screws (replacements in case of damage)**  
Lower Fixation Screws (4)

**NGS -SP-01: Skull Pin Tips (replacements of single use component)**  
MR Compatible Skull Pin Tips


**NGS -SP-02: Pediatric Skull Pin Tips (replacements of single use component)**  
MR Compatible Skull Pin Tips, Pediatric

The Multi-Positional Head Fixation Frame is designed to immobilize the patient's head during the surgical procedure. The patient's head is rigidly attached to the Head Fixation Frame by the physician in the desired position (lateral, oblique, prone or supine) for the planned surgical procedure. The patient's head is positioned inside the GE Flex Coils (optional), and rigidly attached to the Head Fixation Frame.

**CP-MPHFF-332-01 (GE 750w GEMS MR Table Compatible)**


The Head Fixation Frame is compatible with the following MRI Systems:

- GE 750w (including GEMS), 3T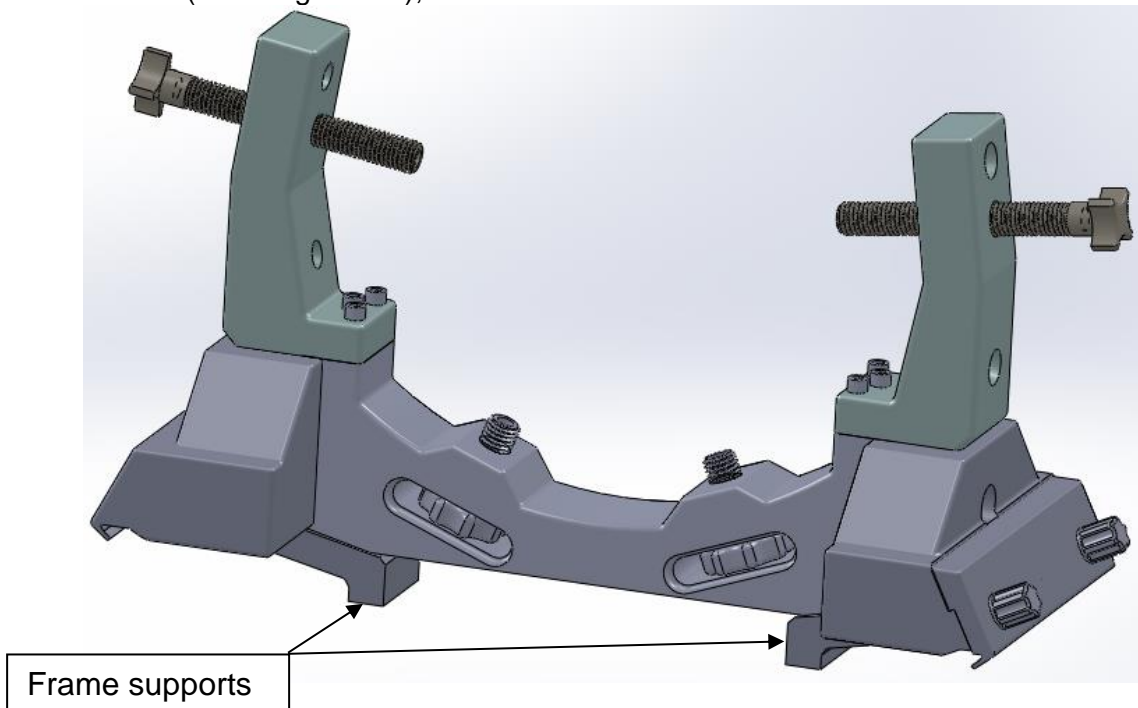

Head Fixation Frame with Table Mount and Frame Supports

- **NOTE:** The Frame Supports are optional. They may be removed by using a 5/16 size hex head wrench to unscrew the screws on the bottom of the table mounts.
- **NOTE:** If using the Frame Supports, make sure they are slightly loose. Do not tighten them until they are tight against the Table Mounts or Frame.
- **CAUTION:** Remove the Frame Supports outside of the scanner room or use only MR Safe hex head wrench. The Frame Supports are completely non-magnetic, but any tool used to remove them may not be. It is best to remove them outside of the scanner room.
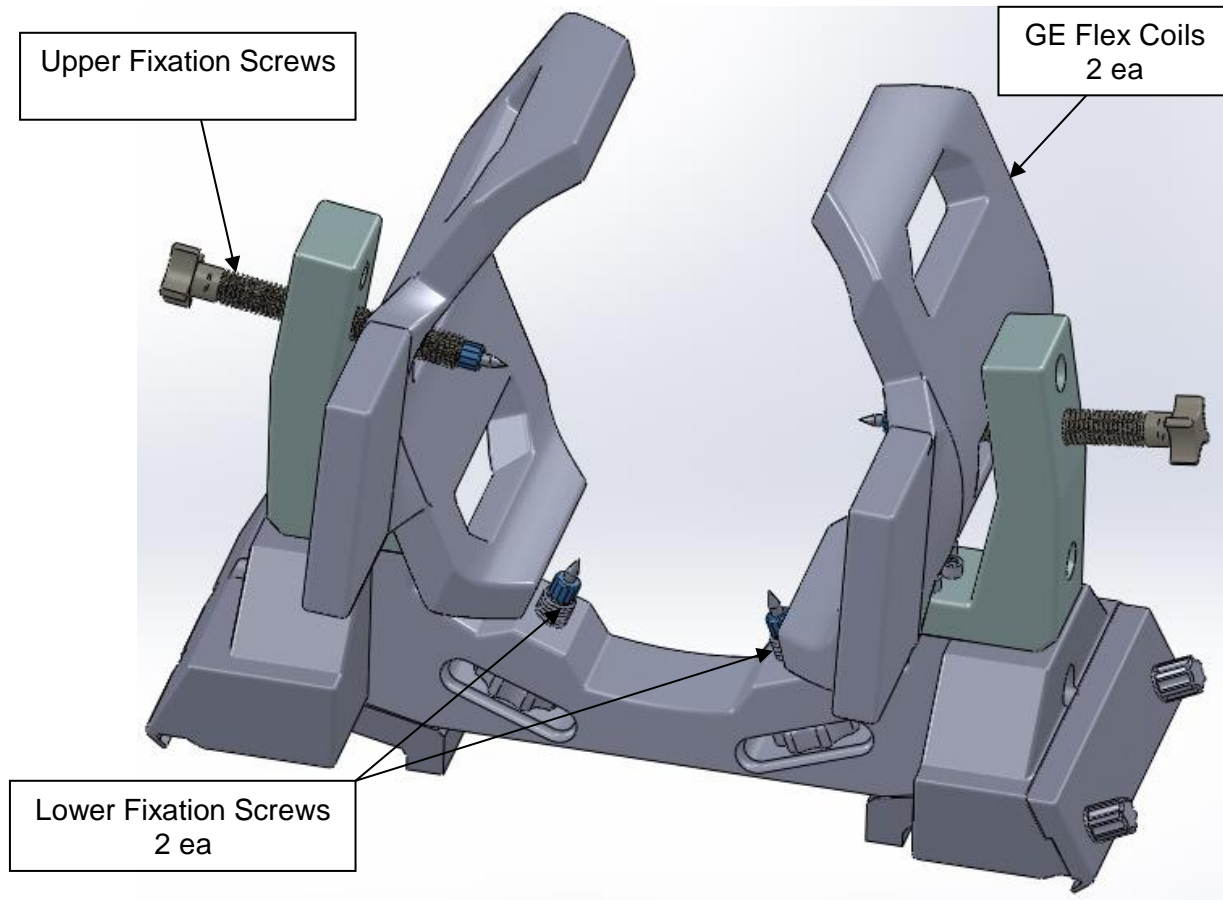


Head Fixation Frame with Fixation Screws and **optional** GE Flex Coils


## 2. Use Instructions

### A. Head Fixation Frame Installation


1. The cradle has a groove on each side running the full length of the cradle.
2. At the superior end of the cradle there are cut-outs on each side.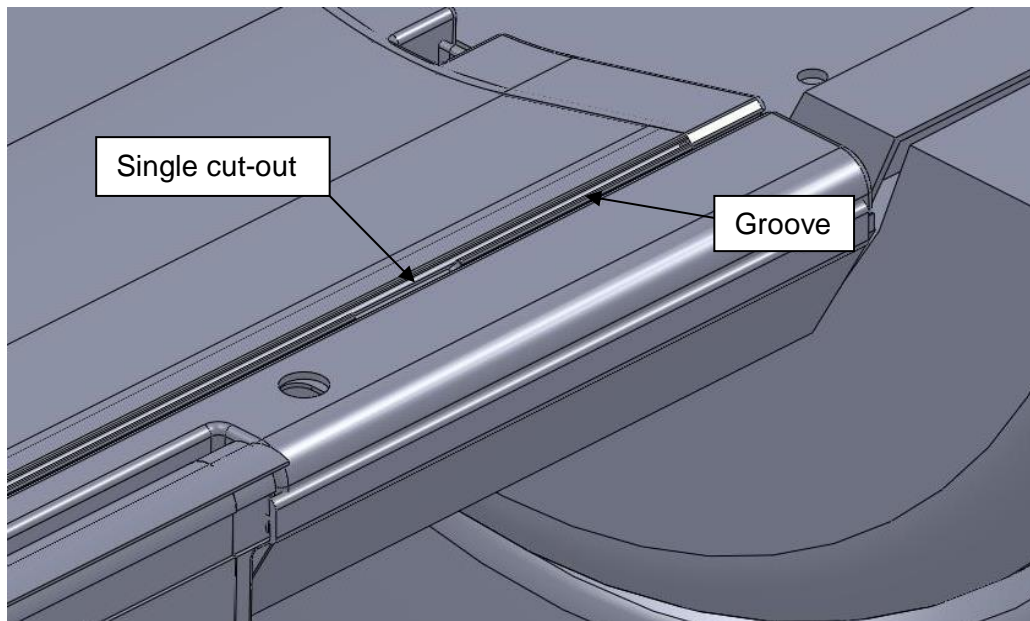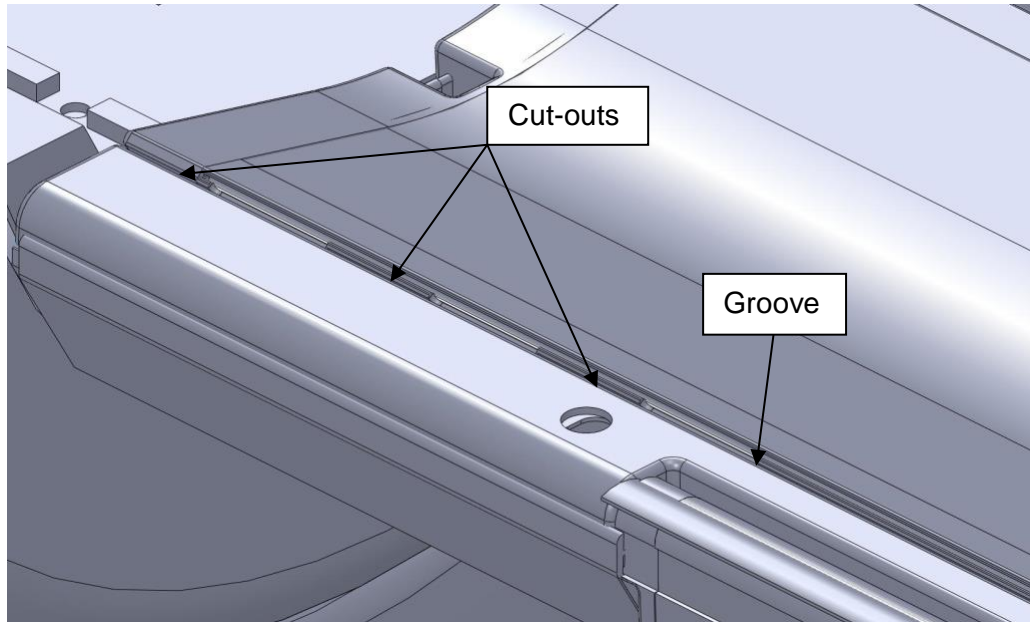


3. Loosen the two (2) Thumb Screws of the Head Fixation Frame. Remove the Adjustable Tab and the Screws.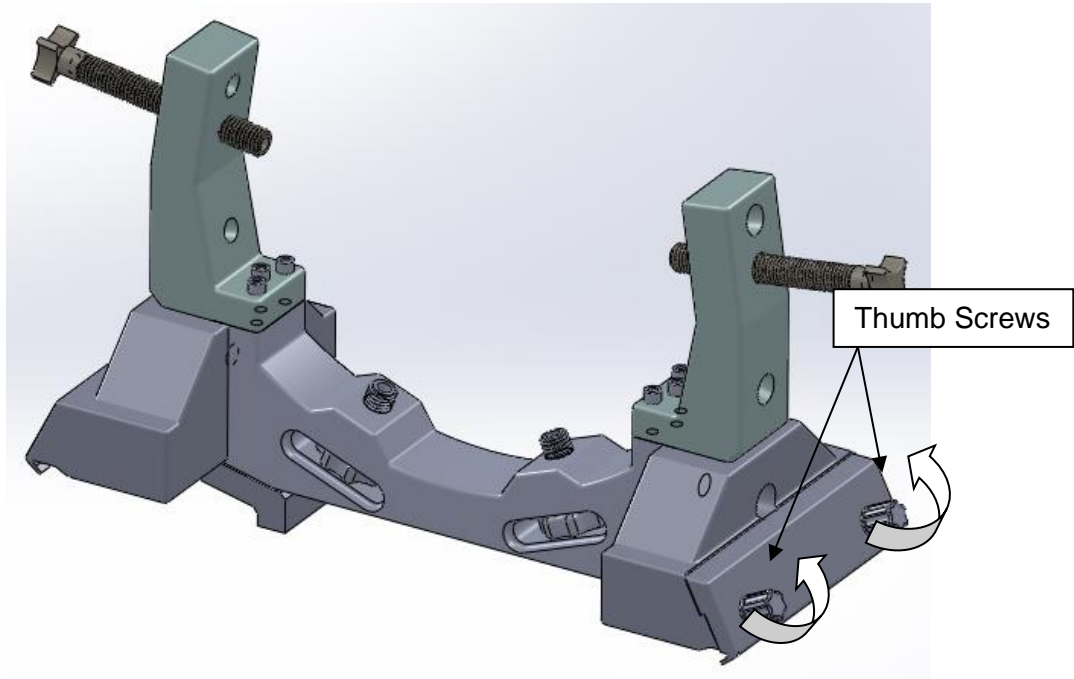


4. The Fixed Side has an engagement "foot" inside the angled tab, as shown below.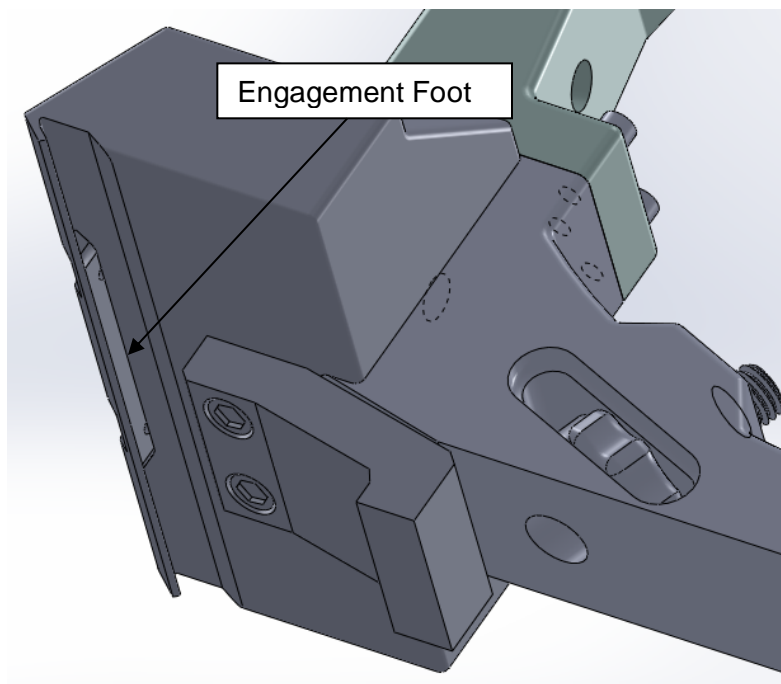

5. Install the lip of the fixed side (no Thumb Screws) of the Table Mount into the groove so the engagement foot is inside the cut-out.

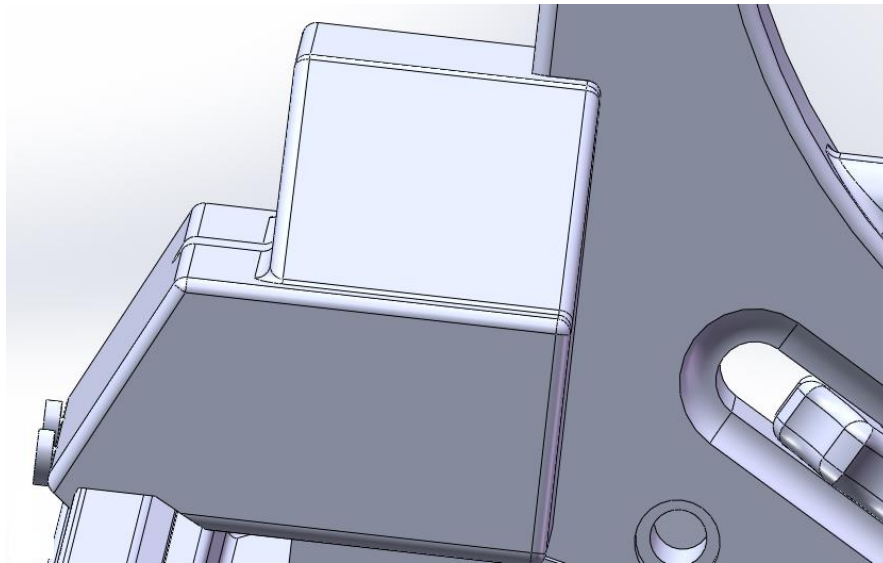
6. Install the lip of the adjustable side Tab (with Thumb Screws) of the Head Fixation Frame into the groove such that the engagement foot fits into the cut-out.

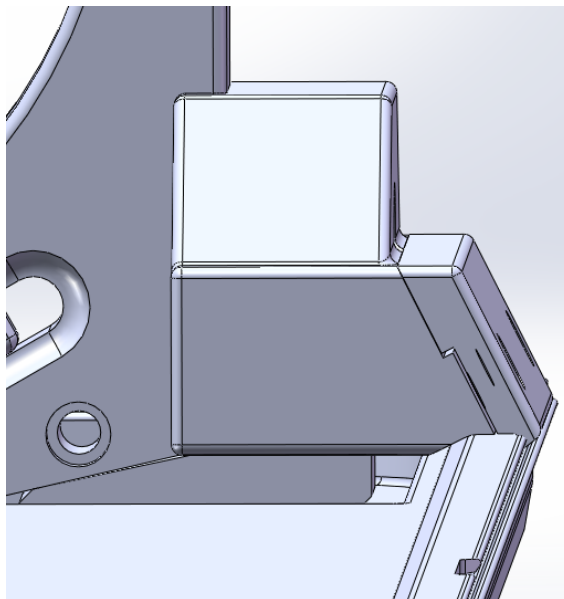7. Apply pressure (squeeze together) the Stabilizer and the Tab until they snap together, insert the Thumb Screws, and then tighten the Thumb Screws.

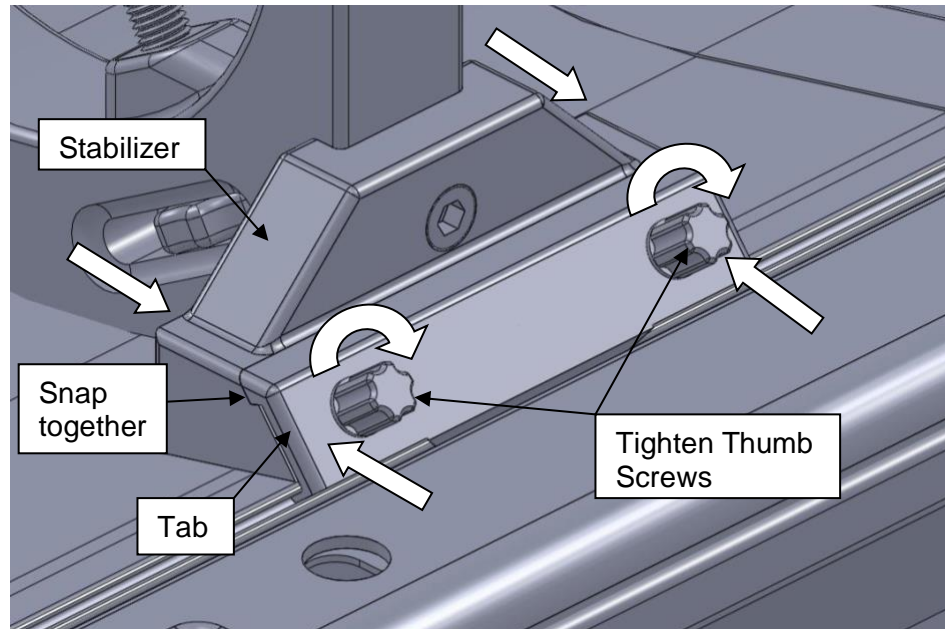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ClearPoint Neuro, Inc.