CLEARPOINT[®]

NEURO

GE-HDx Multi-Positional Head Fixation Frame

INSTRUCTIONS FOR USE - INSTALLATION

NOTE: These installation instructions are to be used in conjunction with the Universal Multi-Positional Head Fixation Frame Instructions for Use.

CE

Table of Contents

1. Device Description
2. Use Instructions
 - A. Head Fixation Frame Installation

| | |
|--|--|
|  | NOTE: These installation instructions are to be used in conjunction with the Universal Multi-Positional Head Fixation Frame Instructions for Use. |
|--|--|

1. Device Description

Package Contents:

CP- MPHFF-312-01: ClearPoint Multi-Positional Head Fixation Frame (New Build)
(GE HDx - MR Table Compatible)

Head Fixation Frame, Upper Fixation Screws (4), Lower Fixation Screws (4), Drive Wheels (2)

-OR-

CP- MPFA-312-01: ClearPoint Multi-Positional Head Fixation Frame (Upgraded)
(GE HDx - MR Table Compatible)

Head Fixation Frame, Upper Fixation Screws (4), Lower Fixation Screws (4), Drive Wheels (2)

Associated Devices:

CP-FFSMPU-01: Upper Fixation Frame Screws (replacements in case of damage)

Upper Fixation Screws (4)

CP-FFSMPL-01: Fixation Frame Screws for GE, Siemens and IMRIS (replacements in case of damage)

Lower Fixation Screws (4)

NGS-SP-01: Skull Pin Tips

MR Compatible Skull Pin Tips


NGS-SP-02: Pediatric Skull Pin Tips

MR Compatible Skull Pin Tips, Pediatric

The Multi-Positional Head Fixation Frame is designed to immobilize the patient's head during the surgical procedure. The patient's head is rigidly attached to the Head Fixation Frame by the physician in the desired position (lateral, oblique, prone or supine) for the planned surgical procedure.

The GE-HDx Multi-Positional Head Fixation Frame is compatible with the following MRI Systems:



- GE-HDx 1.5T and 3T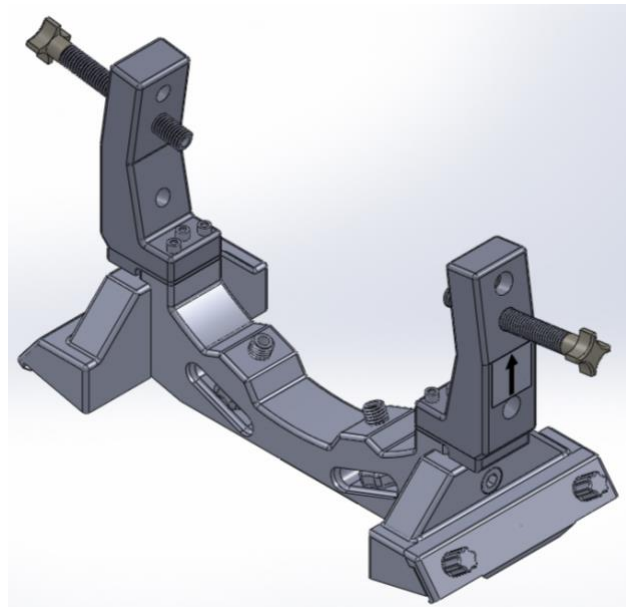


2. Use Instructions

A. Head Fixation Frame Installation


1. The cradle has a groove on each side running the full length of the cradle.
2. At the superior end of the cradle there are three cut-outs on one side and a single cut-out on the other side.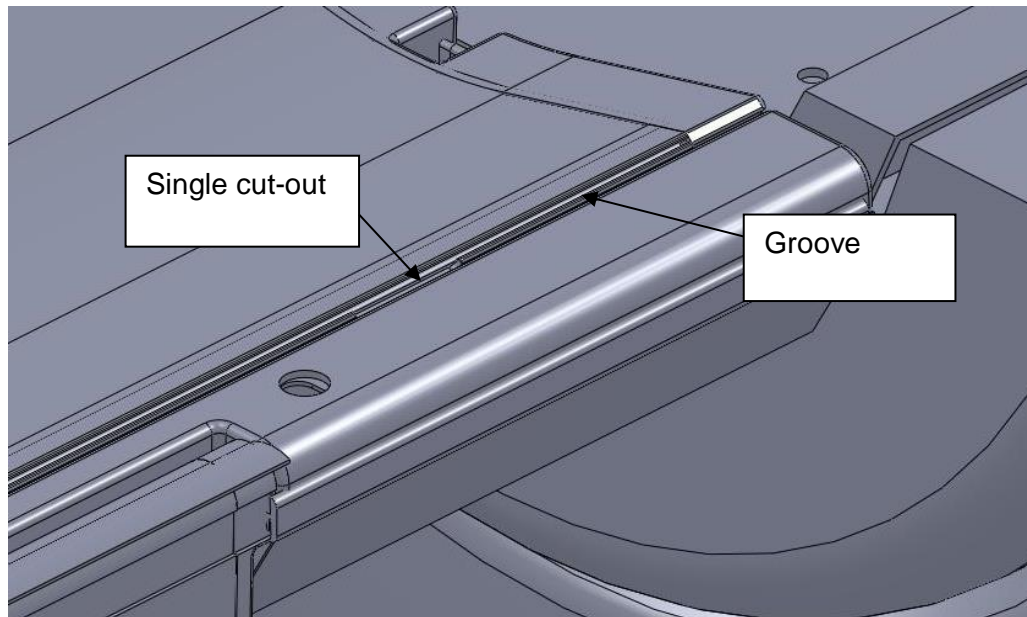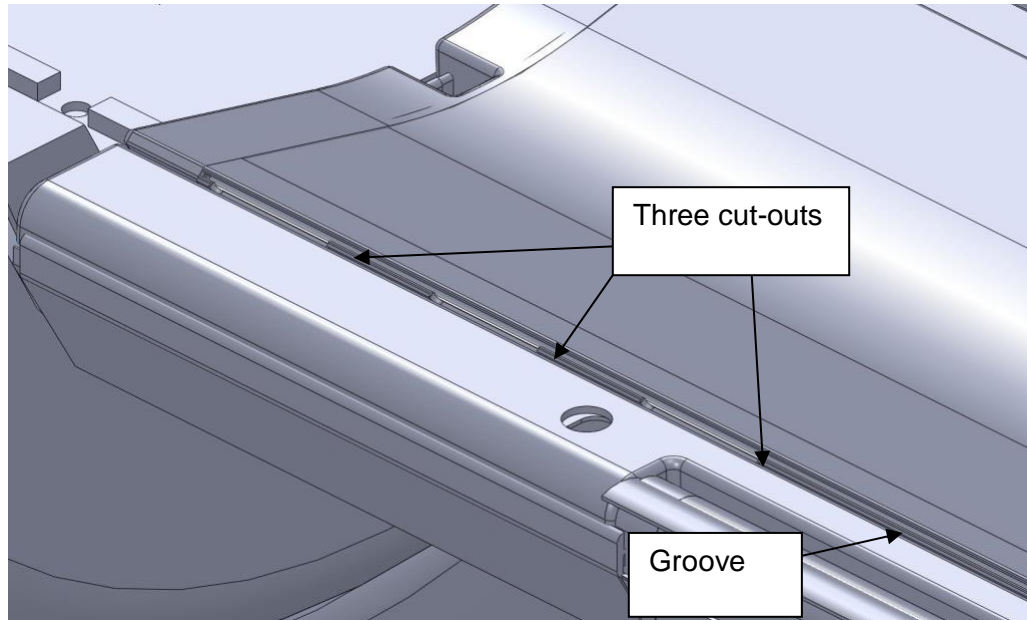


3. Loosen the two (2) Thumb Screws of the Head Fixation Frame.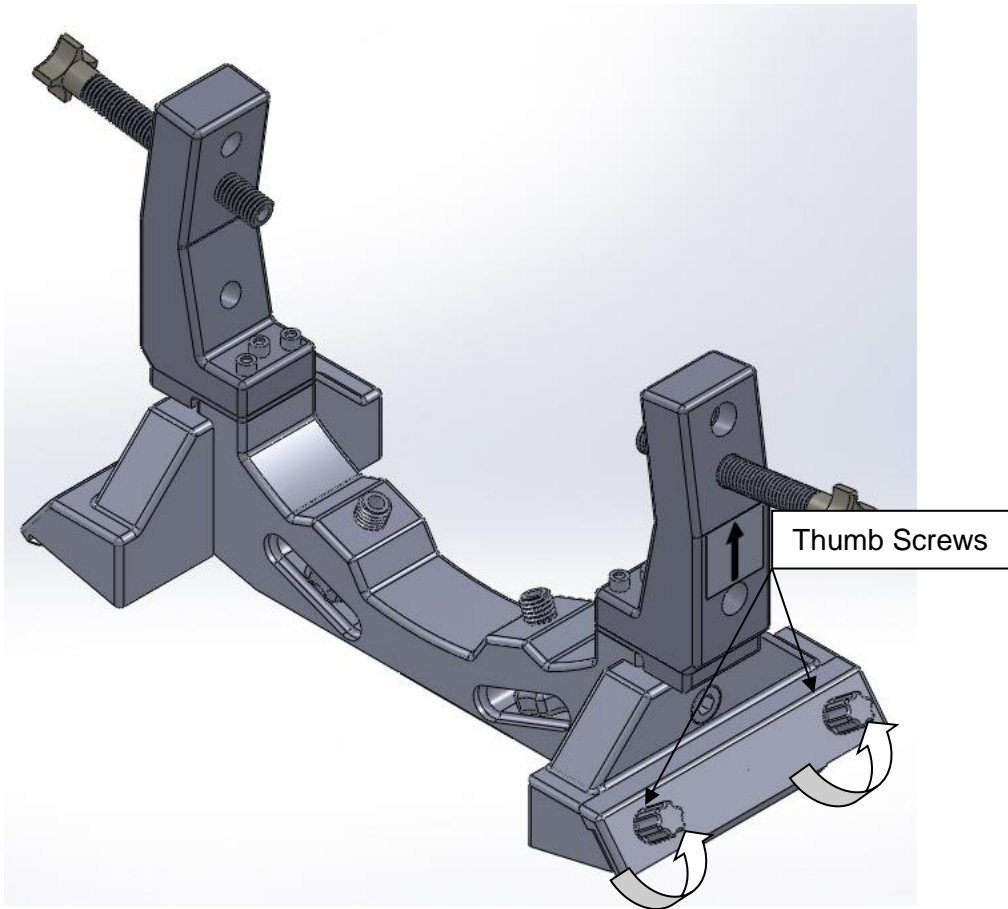


4. Install the Lip of the Fixed Side (no Thumb Screws) of the Head Fixation Frame into the groove at the middle of the three cut-outs with the Toe centered in the middle of the cut-out.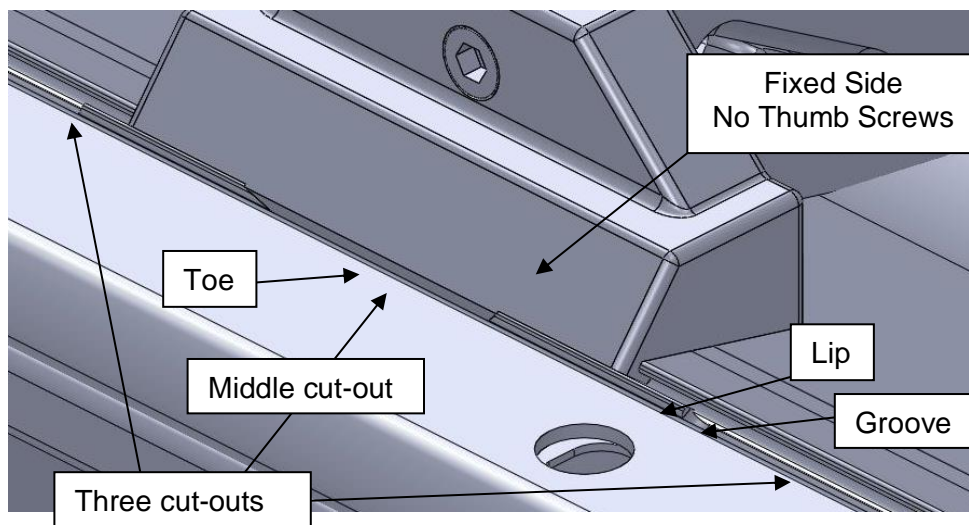

5. Install the lip of the adjustable side Tab (with Thumb Screws) of the Head Fixation Frame into the groove at the single cut-out with the toe centered in the middle of the cut-out.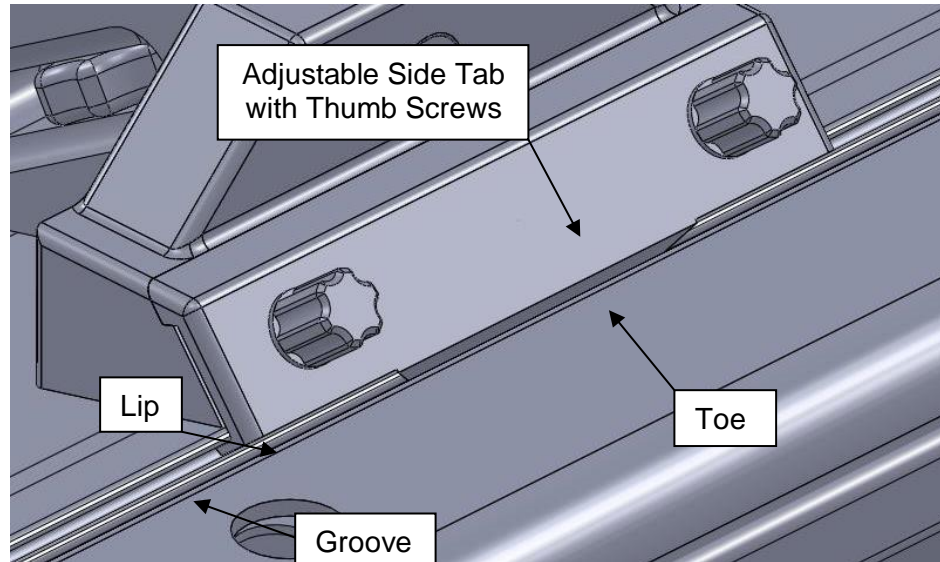

6. Apply pressure (squeeze together) the Stabilizer and the Tab until they snap together and then tighten the Thumb Screws.

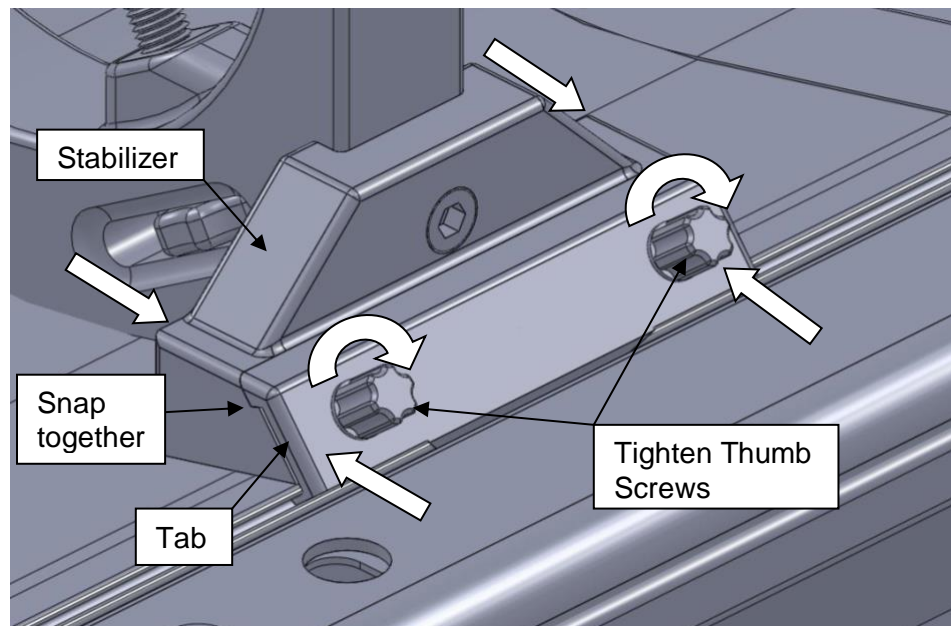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ClearPoint Neuro, Inc.