# CLEARPOINT<sup>®</sup>

## NEURO

**IMRIS with Base Multi-Positional Head Fixation Frame**

**INSTRUCTIONS FOR USE - INSTALLATION**

**NOTE:** These installation instructions are to be used in conjunction with the Universal Multi-Positional Head Fixation Frame Instructions for Use.

## Table of Contents

1. **Device Description**
2. **Use Instructions**
  - A. **Head Fixation Frame Installation**

	<b>NOTE: These installation instructions are to be used in conjunction with the Universal Multi-Positional Head Fixation Frame Instructions for Use.</b>
---	--

## 1. Device Description

### Package Contents:

**CP-MPHFF-211-01: ClearPoint Multi-Positional Head Fixation Frame (New Build)**

(IMRIS ORT 100 & ORT 200 - MR Table Compatible)

Head Fixation Frame, Upper Fixation Screws (4), Lower Fixation Screws (4), Drive Wheels (2)

### Associated Devices:

**CP-FFSMPU-01: Upper Fixation Frame Screws (replacements in case of damage)**

Upper Fixation Screws (4)

**CP-FFSMPL-01: Fixation Frame Screws for GE, Siemens and IMRIS (replacements in case of damage)**

Lower Fixation Screws (4)

**NGS-SP-01: Skull Pin Tips**

MR Compatible Skull Pin Tips


**NGS-SP-02: Pediatric Skull Pin Tips**

MR Compatible Skull Pin Tips, Pediatric

The Multi-Positional Head Fixation Frame is designed to immobilize the patient's head during the surgical procedure. The patient's head is rigidly attached to the Head Fixation Frame by the physician in the desired position (lateral, oblique, prone or supine) for the planned surgical procedure.

The IMRIS Multi-Positional Head Fixation Frame is compatible with the following MRI Systems:


- IMRIS 1.5T and 3T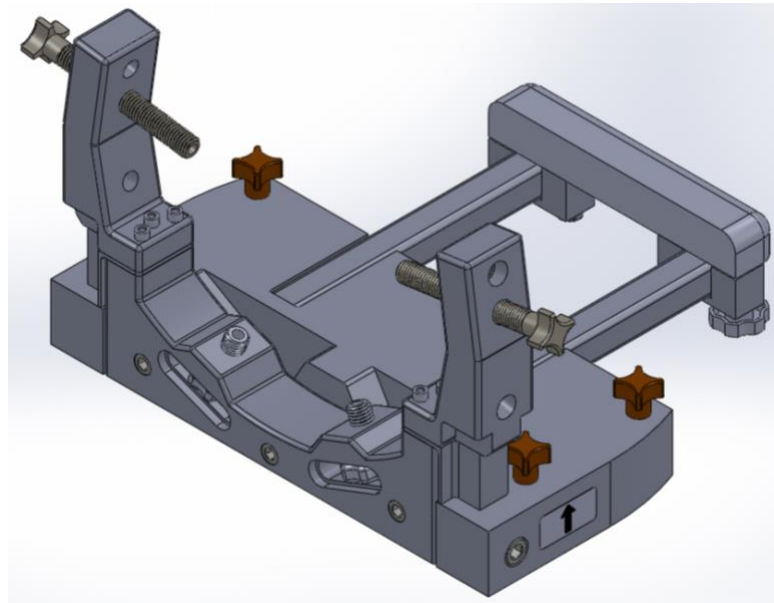


## 2. Use Instructions

### A. Head Fixation Frame Installation


1. The End Table (MR II End Table shown) has 4 ea threaded holes on the top.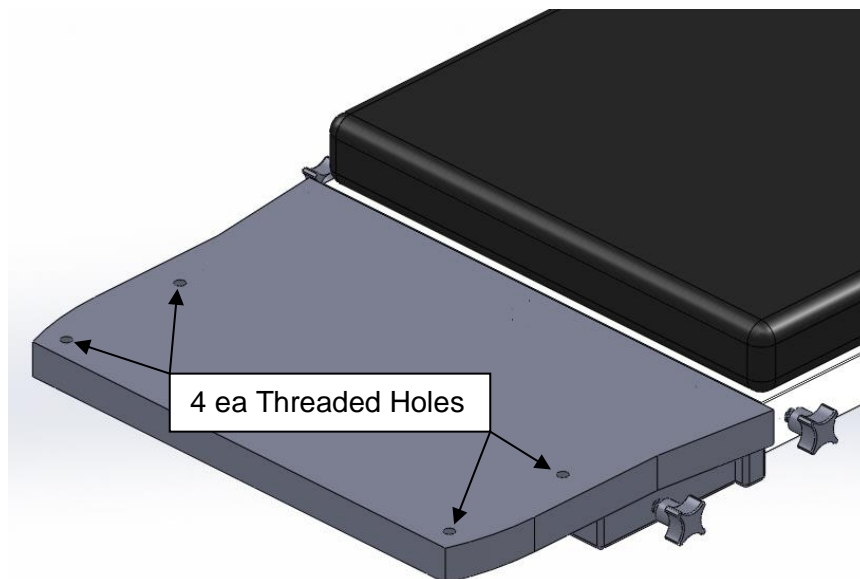

2. Align the 4 ea Thumbscrews with the holes in the End Table.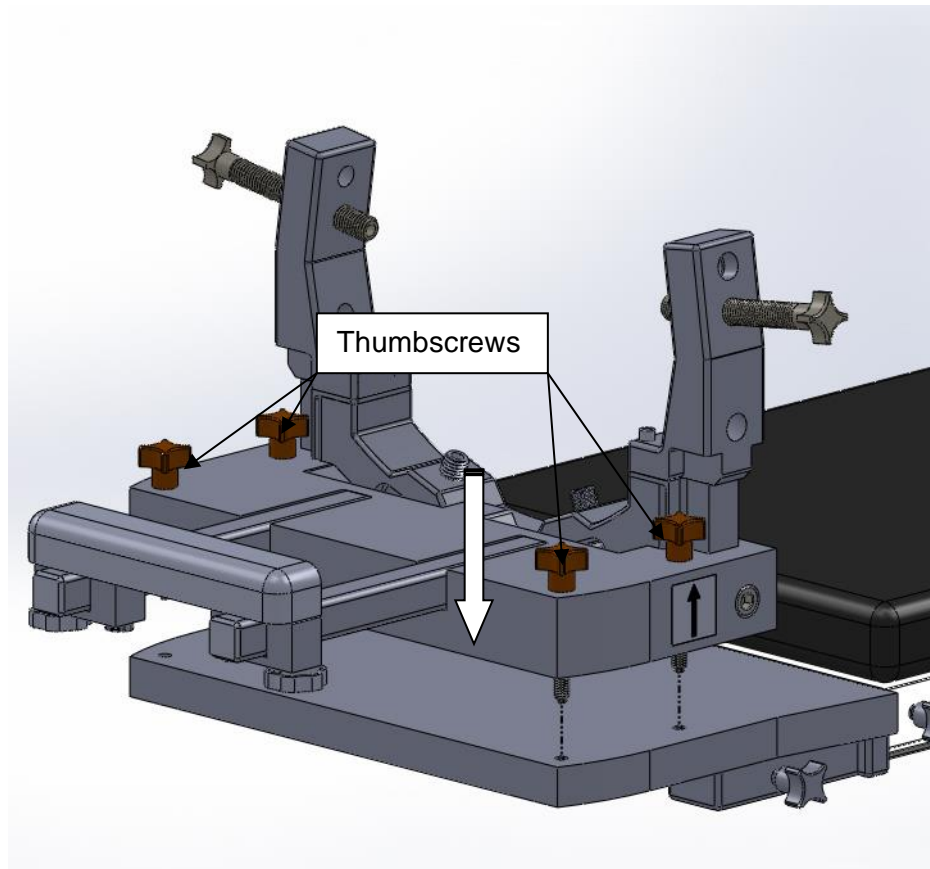

3. Fasten the Head Fixation Frame to the End Table by tightening the Thumbscrews.

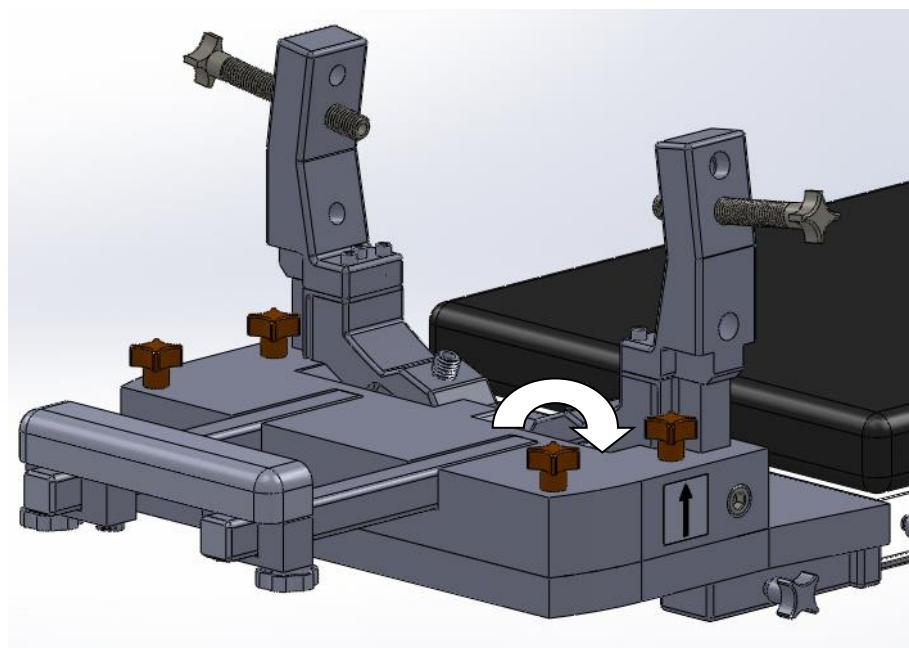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ClearPoint Neuro, Inc.