CLEARPOINT

NEURO

Inflexion Aera-Skyra Head Fixation Frame - BR

INSTRUCTIONS FOR USE - INSTALLATION

Table of Contents

1. Device Description
2. Use Instructions
 - A. Head Fixation Frame Installation

	<p>NOTE: These installation instructions are to be used in conjunction with the Universal Inflexion Head Fixation Frame Instructions for Use</p>
-----------------------------------------------------------------------------------	---------------------------------------------------------------------------------------------------------------------------------------------------------

1. Device Description

Package Contents:

CP-INHFF-122-01-BR: ClearPoint Inflexion Aera-Skyra Head Fixation Frame

Inflexion Head Fixation Frame Base, Lower Ring (1), Upper Ring (1) (*optional*), Posts (6), Short Fixation Screws (4), Medium Fixation Screws (4), Long Fixation Screws (4), Extra Long Fixation Screws (4), Post screws (8) Green or Black Knob (4), Yellow Knob (4), Screw Tool (2), Screw Tool Crossbar (2), Hex Knob (2).

Table Configurations

Fits Siemens Aera-Skyra - MR Table configurations.

Associated Devices

NGS-SP-01: Skull Pin Tips

MR Compatible Skull Pin Tips

NGS-SP-02: Pediatric Skull Pin Tips

MR Compatible Skull Pin Tips, Pediatric

Warning: This device is intended for multiple uses. Contents are non-sterile.

Caution: Federal (U.S.) law restricts this device to sale by or on the order of a physician.

Note: The Head Fixation Frame is **MR Conditional**

The Inflexion Head Fixation Frame is designed to immobilize the patient's head during the surgical procedure. The patient's head is rigidly attached to the Head Fixation Frame by the physician in the desired position (lateral, oblique, prone or supine) for the planned surgical procedure. Positioning of the head may be performed remotely with the Arc separated from the Base or with the Arc attached to the Base.

The Inflexion Aera-Skyra Head Fixation Frame (**see Figure 1**) is compatible with the following MRI Systems:

- SMS – Aera-Skyra 1.5T and 3T

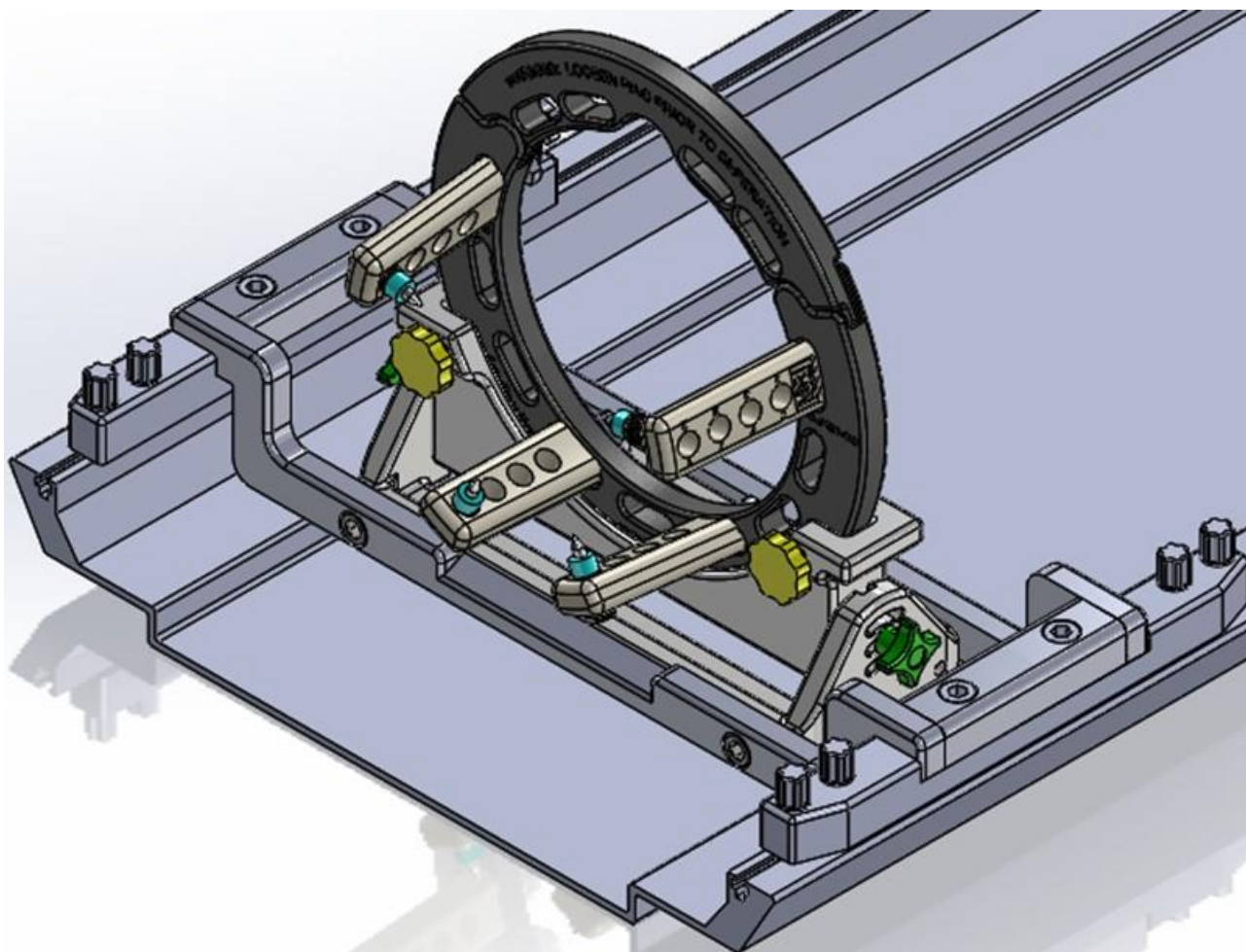
Figure 1: Head Fixation Frame with Table Mounts

2. Use Instructions

A. Head Fixation Frame Installation

1. The cradle has a "T" shaped groove on each side (**Figure 2**). There are also cut outs that the T-Screws will fit in.

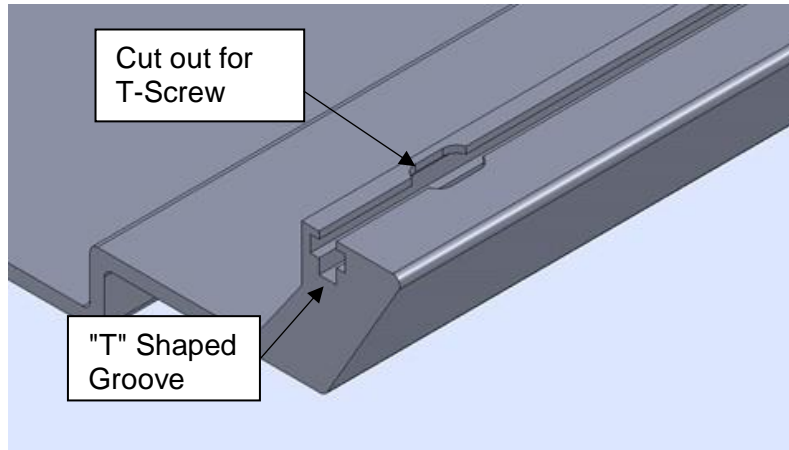
Figure 2

2. Remove the 8 ea T-Screws and place them down into the "T" groove aligning them with the 2 ea T-bars (one on each side). See **Figure 3**.

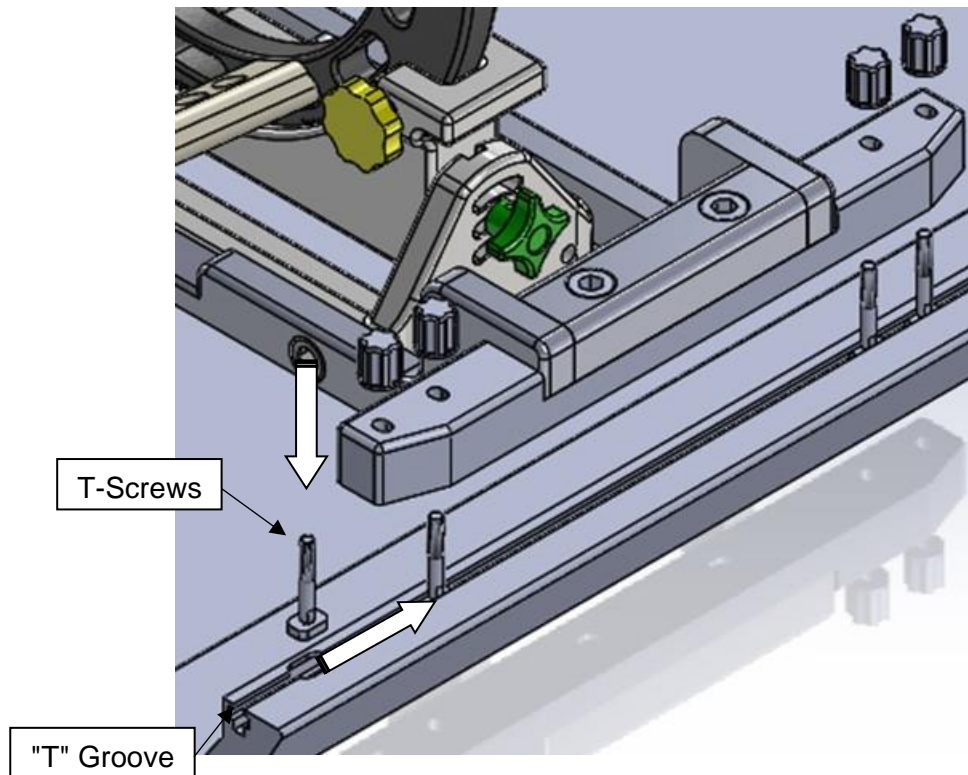
Figure 3

3. Once all 8 ea T-screws are aligned with the 2 ea T-bars – Lower the Head Frame down onto the Cradle. **See Figure 4.**

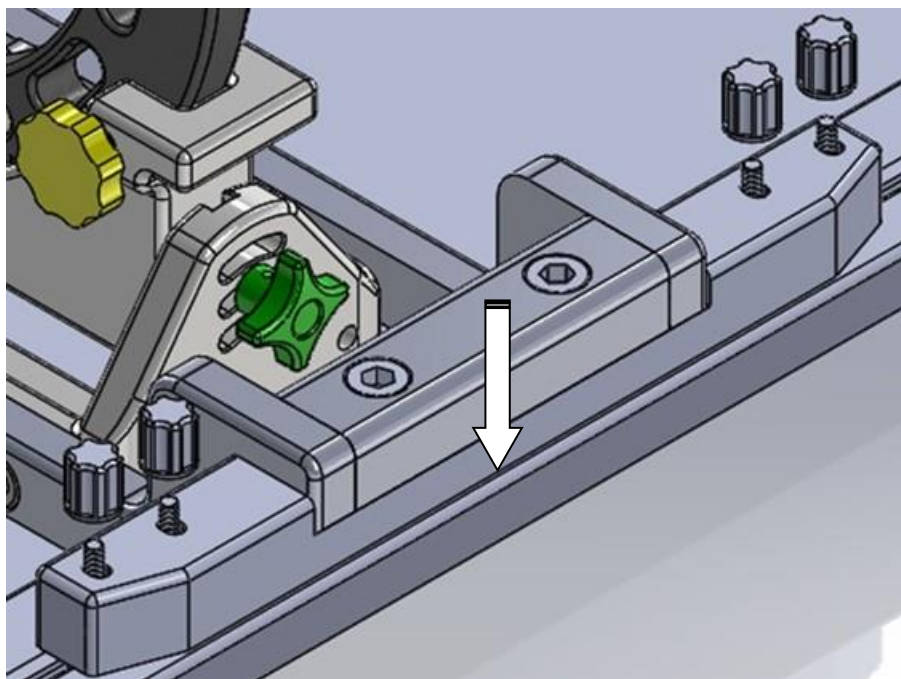
Figure 4

4. Fasten the Head Frame to the Cradle using the 8ea Knobs. **See Figure 5. DO NOT OVER TIGHTEN**

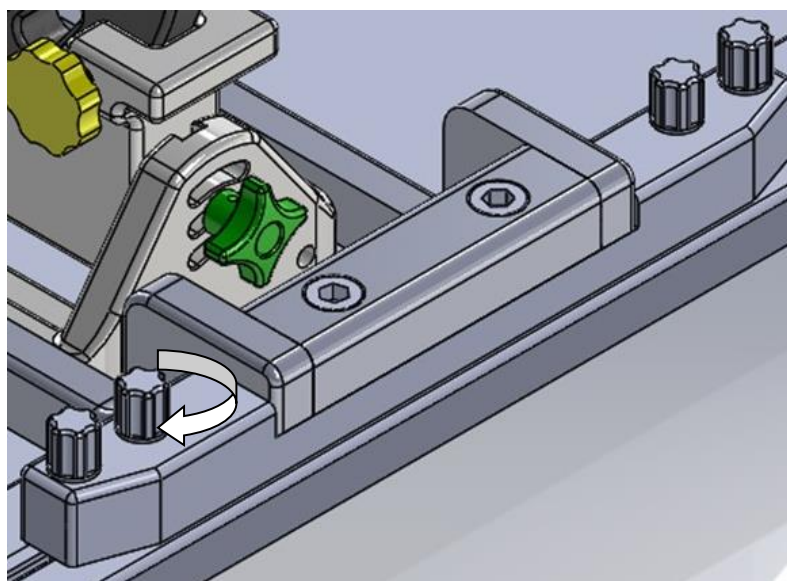
Figure 5

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ClearPoint Neuro, Inc