# CLEARPOINT

## NEURO

**Inflexion Avanto Head Fixation Frame - BR**

**INSTRUCTIONS FOR USE - INSTALLATION**

## Table of Contents

- 1. Device Description
- 2. Use Instructions
  - A. Head Fixation Frame Installation

	<p><b>NOTE:</b> These installation instructions are to be used in conjunction with the Universal Inflexion Head Fixation Frame Instructions for Use</p>
---	---

1. Device Description

Package Contents:

**CP-INHFF-112-01-BR: Inflexion Avanto Head Fixation Frame**

Inflexion Head Fixation Frame Base, Lower Ring (1), Upper Ring (1) *(optional)*, Posts (6), Short Fixation Screws (4), Medium Fixation Screws (4), Long Fixation Screws (4), Extra Long Fixation Screws (4), Post screws (8) Green or Black Knob (4), Yellow Knob (4), Screw Tool (2), Screw Tool Crossbar (2), Hex Knob (2).

Table Configurations

Fits Siemens Avanto / Espree / Symphony / Sonata / Trio / Verio - MR Table configurations.

Associated Devices

**NGS-SP-01: Skull Pin Tips**

MR Compatible Skull Pin Tips

**NGS-SP-02: Pediatric Skull Pin Tips**

MR Compatible Skull Pin Tips, Pediatric

**Warning:** This device is intended for multiple uses. Contents are non-sterile.

**Caution:** Federal (U.S.) law restricts this device to sale by or on the order of a physician.

**Note:** The Head Fixation Frame is **MR Conditional**

The Inflexion Head Fixation Frame is designed to immobilize the patient's head during the surgical procedure. The patient's head is rigidly attached to the Head Fixation Frame by the physician in the desired position (lateral, oblique, prone or supine) for the planned surgical procedure. Positioning of the head may be performed remotely with the Arc separated from the Base or with the Arc attached to the Base.

This Inflexion Avanto Head Fixation Frame is compatible with the following MRI Systems: Siemens 1.5T and 3T (Avanto / Espree / Symphony / Sonata / Trio / Verio). **See Figure 1 and Figure 2.**

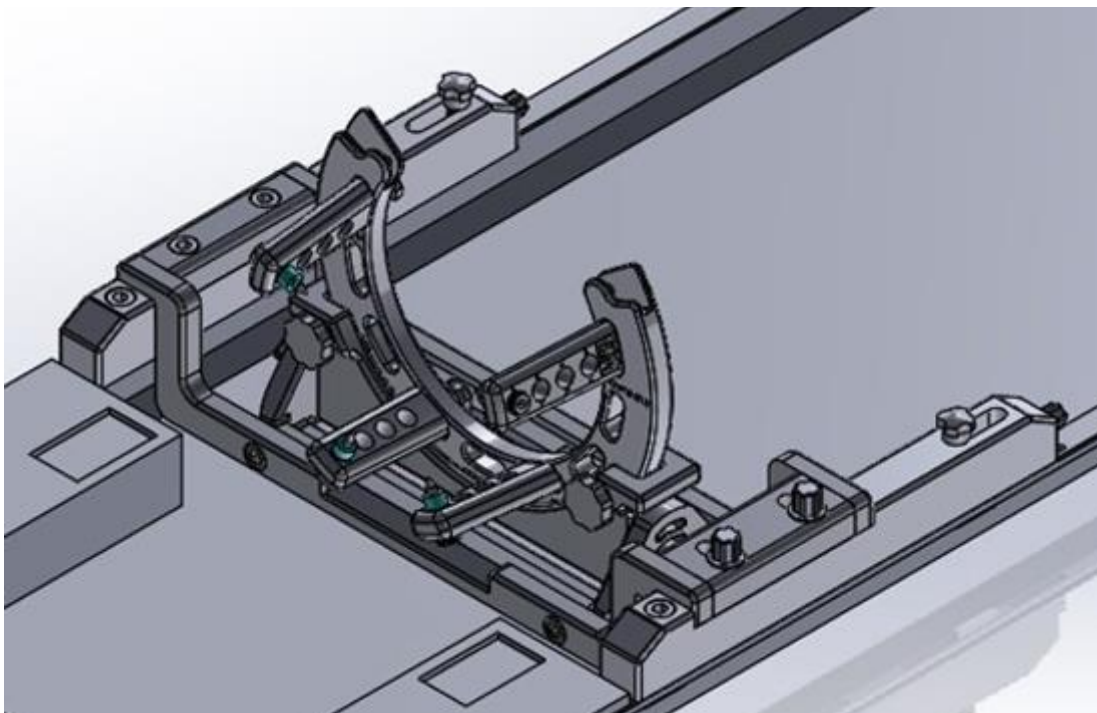
Figure 1: Head Fixation Frame with Table Mounts

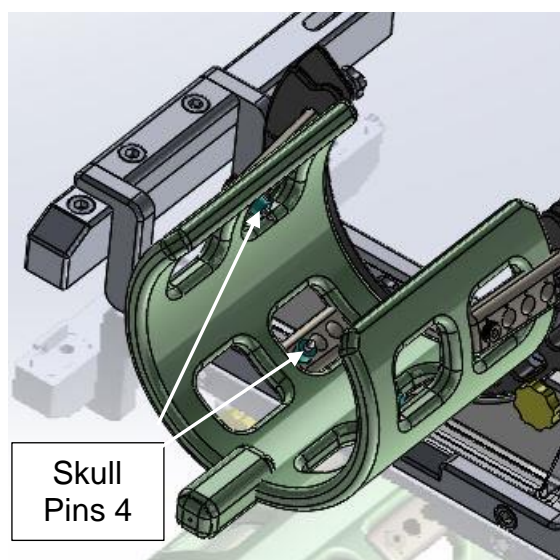
Figure 2: Head Fixation Frame with *optional* SMS Flex Coil

2. Use Instructions

A Head Fixation Frame Installation

1. The cradle has slotted holes equally spaced on both sides of the cradle. (Figure 3)

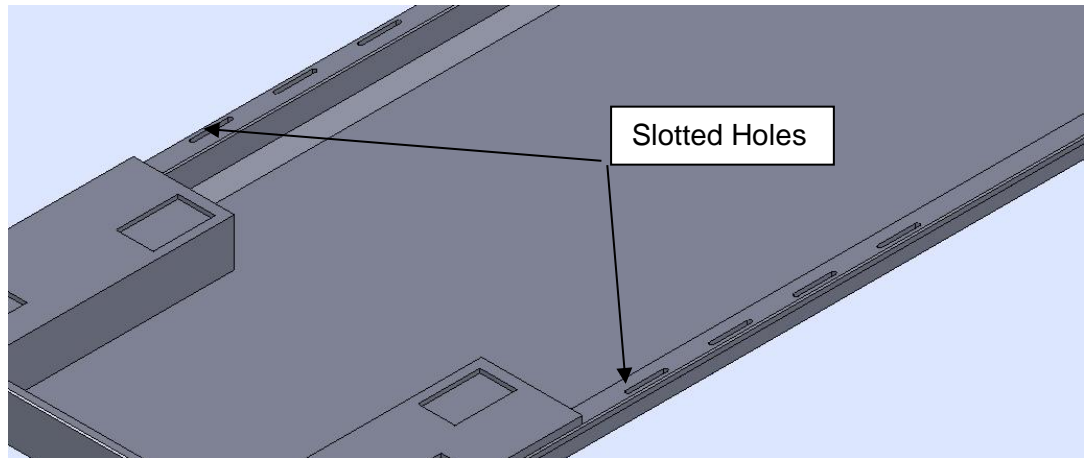
Figure 3

2. Loosen the two width adjustment Thumbscrews and adjust the width of the Head Frame so the Fixed Toe Inserts fit into the slotted holes on both sides of the table. (Figure 4a and 4b)

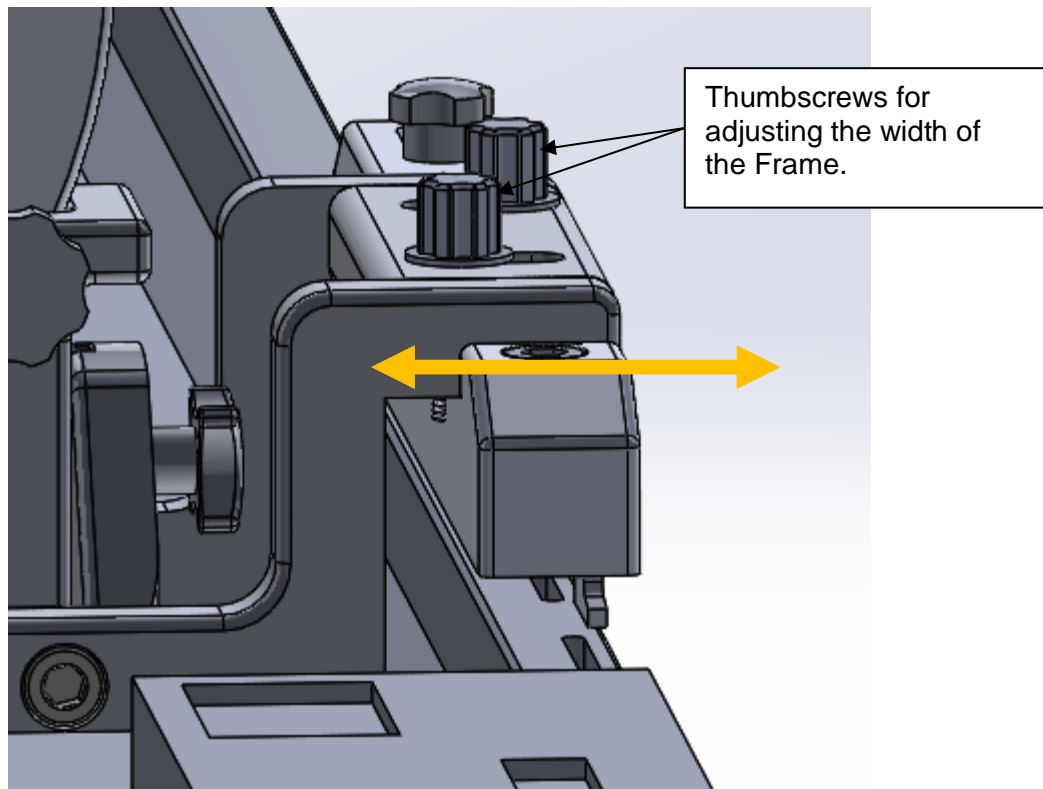
Figure 4a

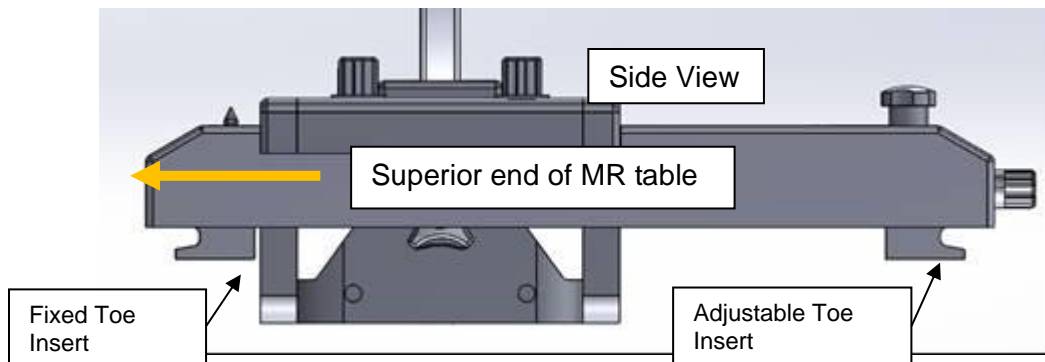
Figure 4b

3. Loosen the Top Thumbscrews, then turn the Front Thumbscrews counterclockwise until the Adjustable Toe Inserts (both sides) are all the way back (Figure 5).

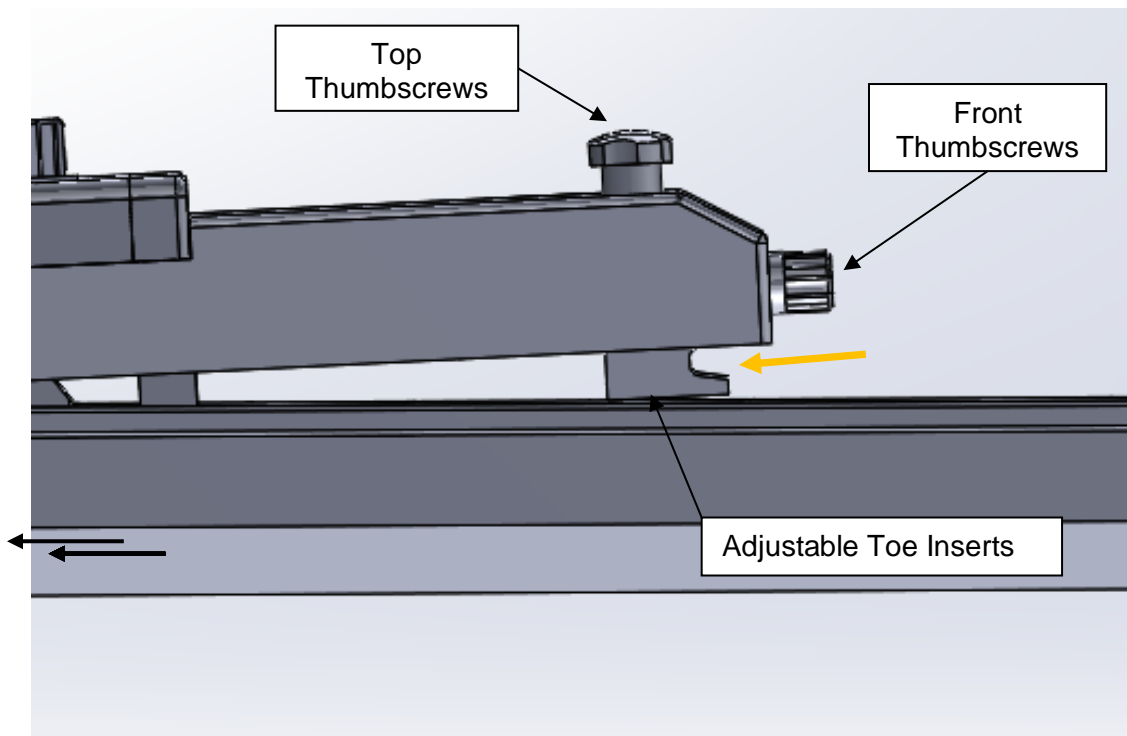
Figure 5

4. Insert the two Fixed Toe Inserts into the slotted holes and then insert the two Adjustable Toe Inserts (Figure 6).  
**Note: If the Toe Inserts will not recede far enough to fit into the Table, remove the front Thumb Screws and use the longer Thumb Screws provided.**

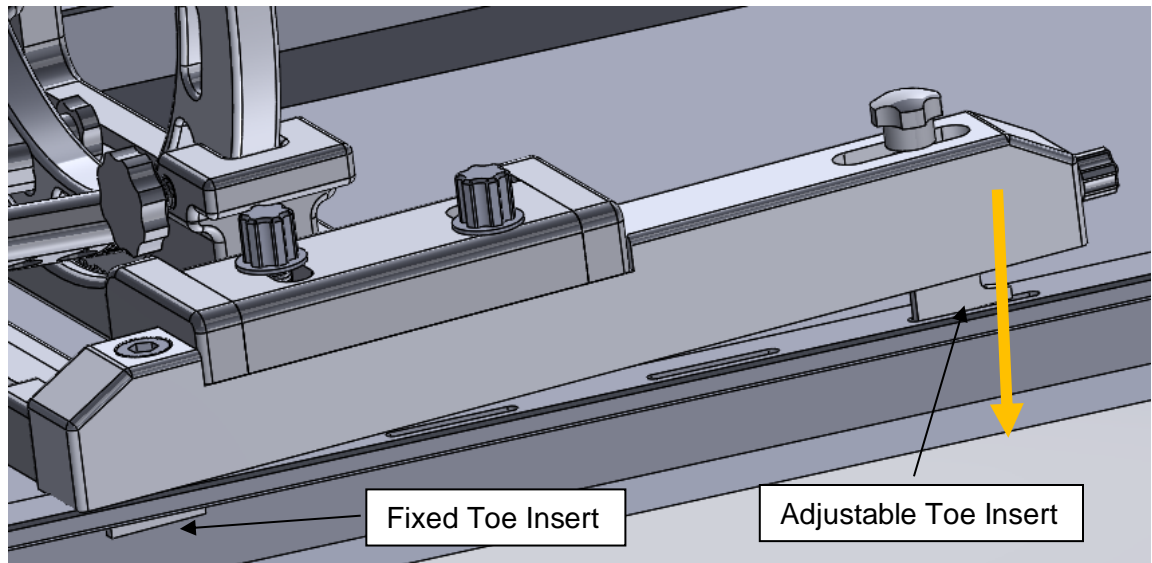
Figure 6

5. Tighten the front two Thumbscrews until the Adjustable Toe Inserts are snug and then tighten the top two Thumbscrews to lock the Toe inserts into place. Finally tighten the two width adjustment Thumbscrews to secure the Head Frame to the table (Figure 7).

**Caution:** Over tightening the front two Thumbscrews may result in breaking the Toe Inserts.

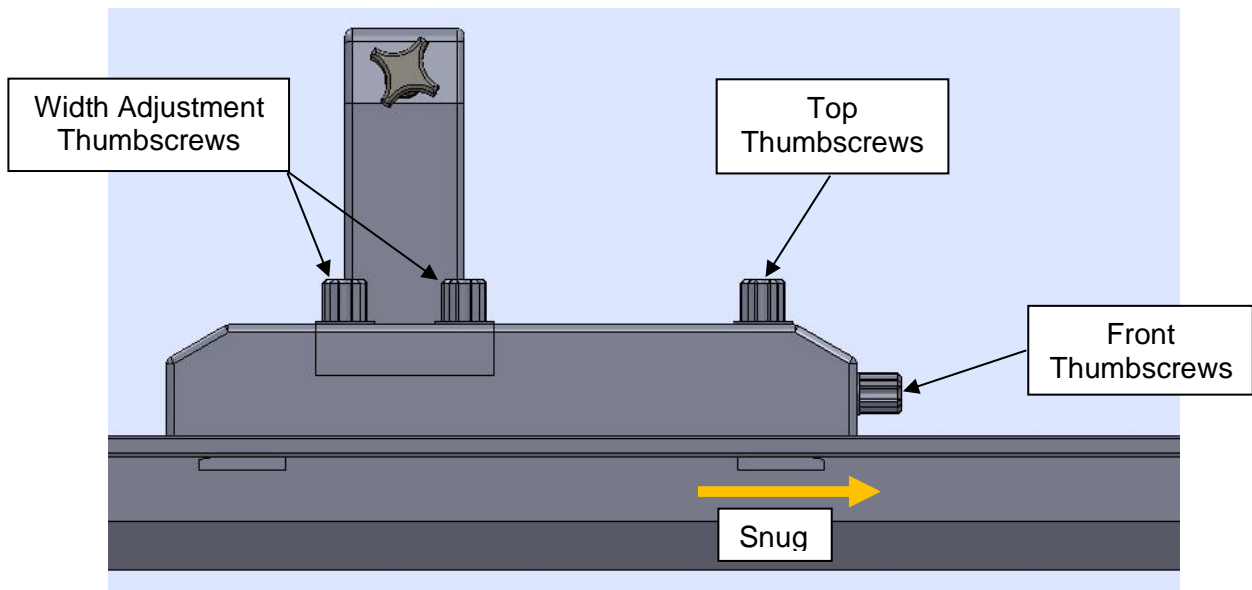
Figure 7

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ClearPoint Neuro, Inc