CLEARPOINT

NEURO

Inflexion GE Premier Head Fixation Frame - BR

INSTRUCTIONS FOR USE - INSTALLATION

Table of Contents

- 1. **Device Description**
- 2. **Use Instructions**
 - A. **Head Fixation Frame Installation**

	<p>NOTE: These installation instructions are to be used in conjunction with the Universal Inflexion Head Fixation Frame Instructions for Use</p>
---	---

1. Device Description

Package Contents:

CP-INHFF-342-01-BR: ClearPoint Inflexion GE Premier Head Fixation Frame

Inflexion Head Fixation Frame Base, Lower Ring (1), Upper Ring (1) (*optional*), Posts (6), Short Fixation Screws (4), Medium Fixation Screws (4), Long Fixation Screws (4), Extra Long Fixation Screws (4), Post screws (8), Green or Black Knob (4), Yellow Knob (4), Screw Tool (2), Screw Tool Crossbar (2), Hex Knob (2).

Table Configurations

Fits GE Premier - MR Table configurations.

Associated Devices

NGS-SP-01: Skull Pin Tips

MR Compatible Skull Pin Tips

NGS-SP-02: Pediatric Skull Pin Tips

MR Compatible Skull Pin Tips, Pediatric

Warning: This device is intended for multiple uses. Contents are non-sterile.

Caution: Federal (U.S.) law restricts this device to sale by or on the order of a physician.

Note: The Head Fixation Frame is **MR Conditional**

The Inflexion Head Fixation Frame is designed to immobilize the patient's head during the surgical procedure. The patient's head is rigidly attached to the Head Fixation Frame by the physician in the desired position (lateral, oblique, prone or supine) for the planned surgical procedure. Positioning of the head may be performed remotely with the Arc separated from the Base or with the Arc attached to the Base.

The Inflexion GE Premier Head Fixation Frame is compatible with the following MRI Systems:

- GE Premier 1.5T and 3T. **See Figure 1.**

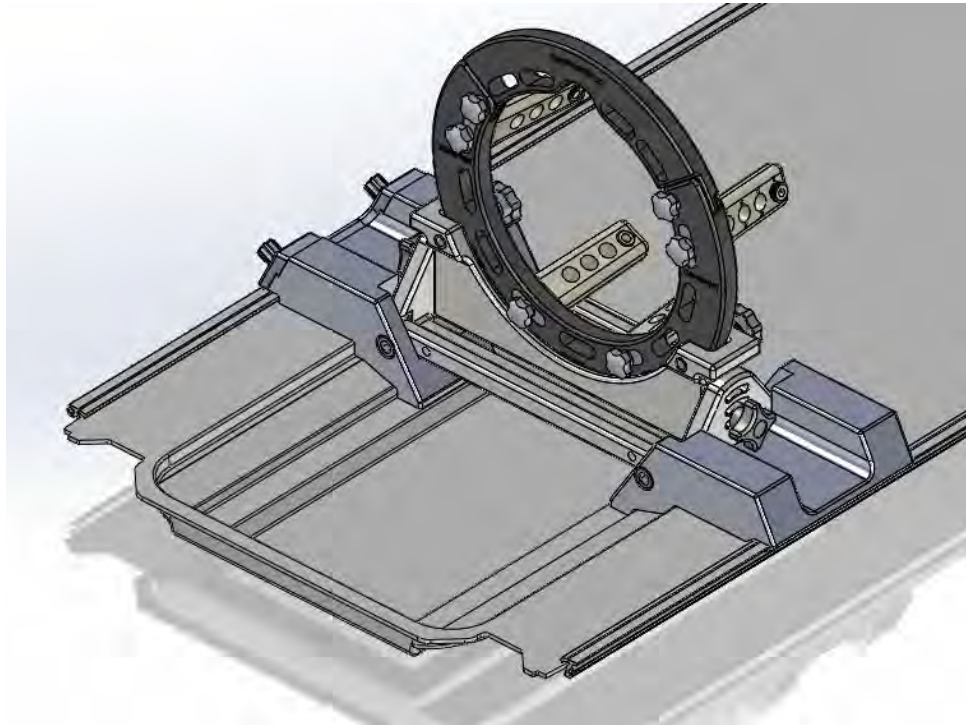
Figure 1: Head Fixation Frame with Table Mounts

2. Use Instructions

A Head Fixation Frame Installation

1. The cradle has a groove on each side running the full length of the cradle.
2. At the superior end of the cradle there is a cut-out on each side equally positioned on the cradle. There is also a "step" near the cut-outs. See Figure 2.

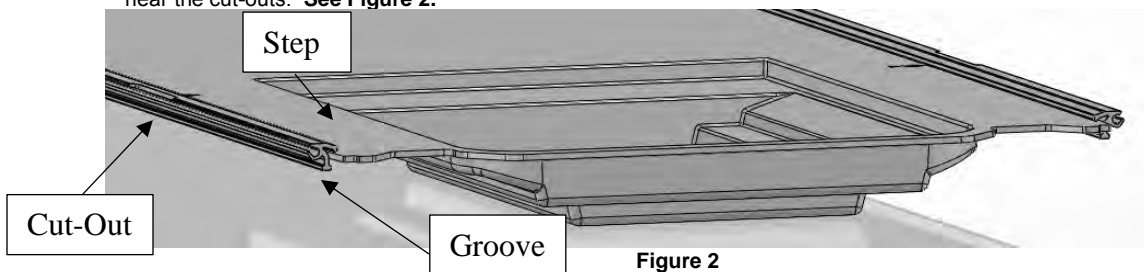
Figure 2

3. Loosen the two (2) Thumb Screws of the Head Fixation Frame. Remove the Adjustable tab and screws. See Figure 3.

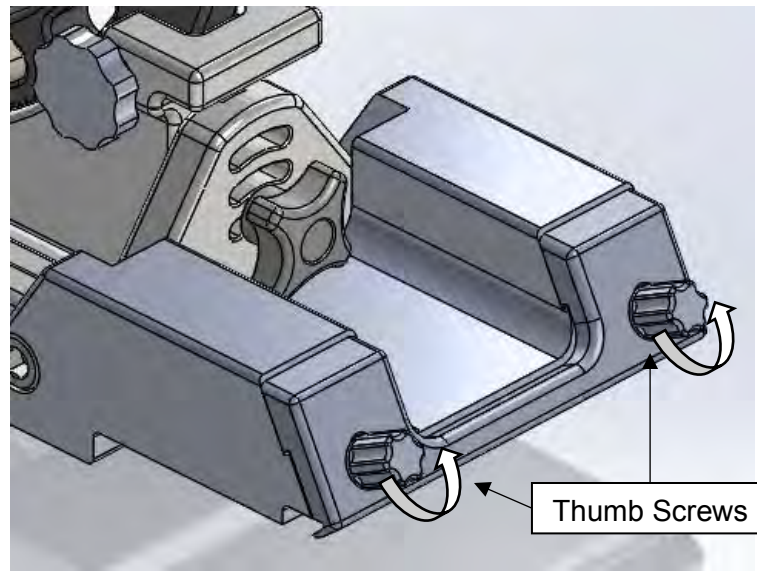
Figure 3

4. The Fixed Side has an engagement “foot” inside the angled tab, as shown in **Figure 4**. Also, both the Fixed and Adjustable sides have a step on the back side of the Table Mounts, as shown in **Figure 5**.

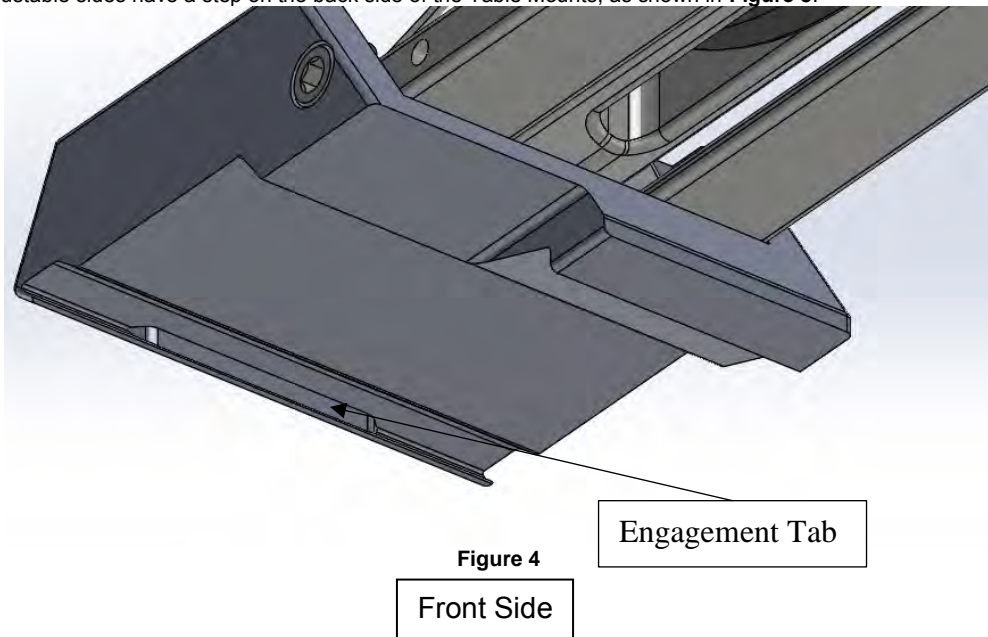
Figure 4

Front Side

Engagement Tab

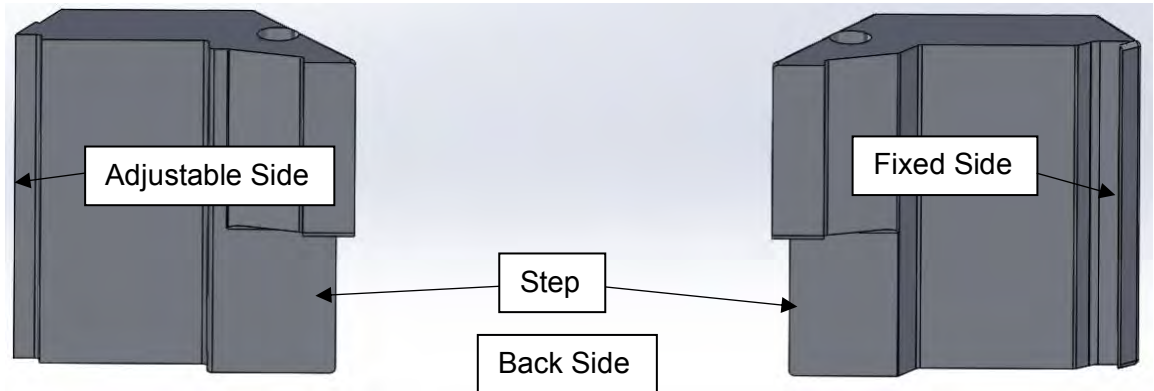
Figure 5

5. Install the lip of the fixed side (no Thumb Screws) of the Table Mount into the groove so the engagement foot is inside the cut-out, as shown in **Figure 6** and the step is distal from the superior end of the table, as shown in **Figure 7**. If the Table Mount will not fit, the assembly needs to be rotated 180 degrees so it fits into the depression in the table.

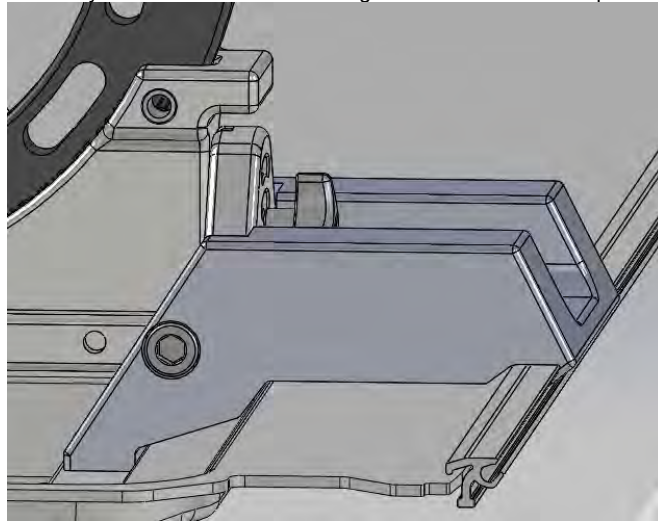
Figure 6

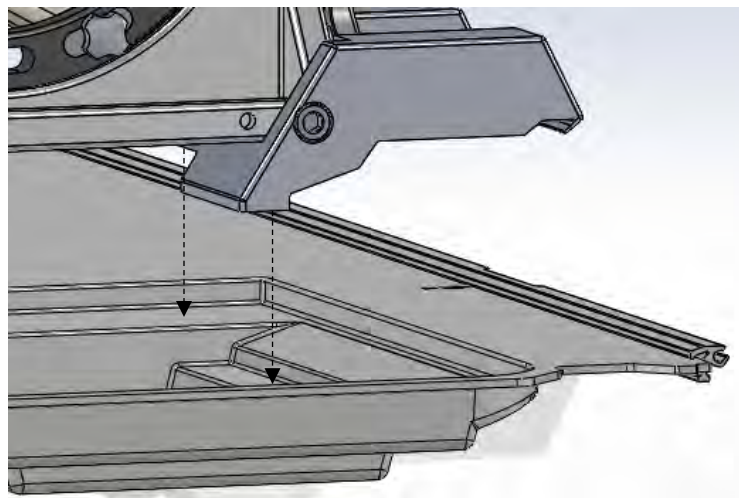
Figure 7

6. Install the lip of the adjustable side Tab (with Thumb Screws) of the Head Fixation Frame into the groove such that the engagement foot fits into the cut-out. **See Figure 8.**

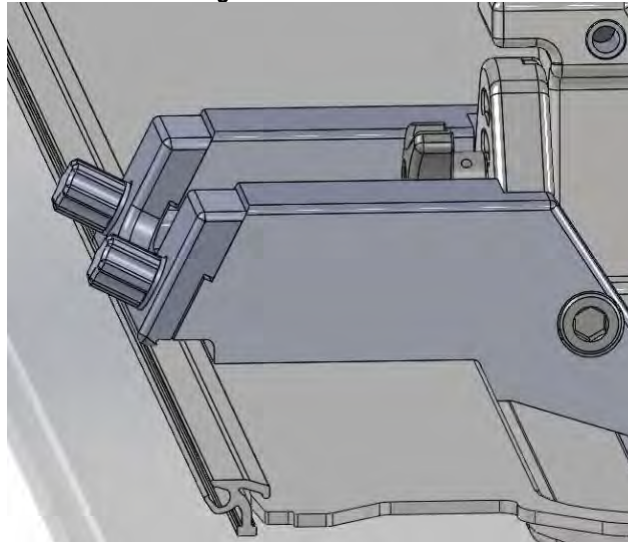
Figure 8

7. Apply pressure (squeeze together) the Table Mount and the Tab until they snap together, insert the Thumb Screws, and then tighten the Thumb Screws. **See Figure 9.**

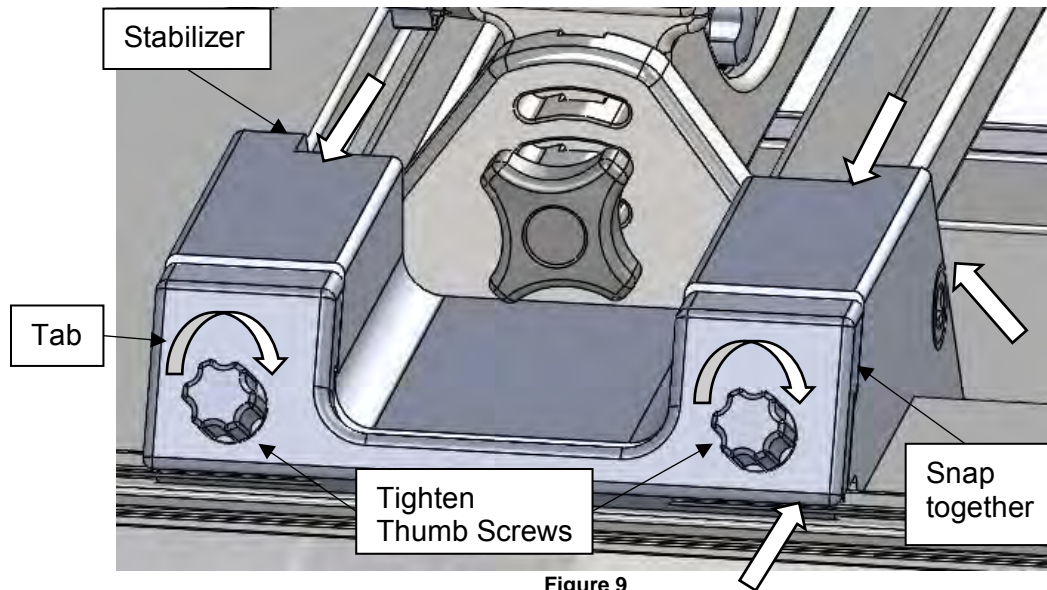
Figure 9

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ClearPoint Neuro, Inc