# CLEARPOINT<sup>®</sup>

## NEURO

**Inflexion Head Fixation Frame IMRIS Aera-Skyra**

**INSTRUCTIONS FOR USE - INSTALLATION**

## Table of Contents

- 1. **Device Description**
- 2. **Use Instructions**
  - A. **ORT 100 – Installation onto the IMRIS Surgical Table**
  - B. **ORT 200 – Installation onto the IMRIS Surgical Table**
  - C. **ORT 400 – Installation onto the IMRIS Surgical Table**

|  |   |
|--|---|
|  | <p><b>NOTE: These installation instructions are to be used in conjunction with the Universal Inflexion Head Fixation Frame Instructions for Use</b></p> |
|--|---|

1. **Device Description**

**Package Contents:**

**CP-INHFF-242-01: Inflexion IMRIS Aera-Skyra Head Fixation Frame**

Inflexion Head Fixation Frame Base, Lower Ring (1), Upper Ring (1) (*optional*), Posts (6), Short Fixation Screws (4), Medium Fixation Screws (4), Long Fixation Screws (4), Extra Long Fixation Screws (4), Post screws (8) Green or Black Knob (4), Yellow Knob (4), Screw Tool (2), Screw Tool Crossbar (2), Hex Knob (2).

**Table Configurations**

Fits Surgical Tables on IMRIS Systems (T-Series/ORT-100, ORT-200, ORT-300, and ORT-400).

**Associated Devices**

**CP-ITE-01: ORT 200 IMRIS Table Extension**

Adaptor for use of ClearPoint Inflexion Head Fixation Frame on ORT 200 and ORT 300 IMRIS tables.

**CP-ITE-02: ORT 400 IMRIS Table Extension**

Adaptor for use of ClearPoint Inflexion Head Fixation Frame on ORT 400 IMRIS tables.

**CP-STB-210-01: CLEARPOINT® IMRIS Table Stabilizer**

Adaptor for use of ClearPoint Inflexion Head Fixation Frame on IMRIS ORT 100, ORT 200, ORT 300 and ORT 400 tables.

**NGS-SP-01: Skull Pin Tips**

MR Compatible Skull Pin Tips, Adult

**NGS-SP-02: Pediatric Skull Pin Tips**

MR Compatible Skull Pin Tips, Pediatric

**Warning:** This device is intended for multiple uses. Contents are non-sterile.


**Caution:** Federal (U.S.) law restricts this device to sale by or on the order of a physician.

**Note:** The Head Fixation Frame is **MR Conditional**

The Inflexion Head Fixation Frame is designed to immobilize the patient's head during the surgical procedure. The patient's head is rigidly attached to the Head Fixation Frame by the physician in the desired position (lateral, oblique, prone or supine) for the planned surgical procedure. Positioning of the head may be performed remotely with the Arc separated from the Base or with the Arc attached to the Base.

The Inflexion IMRIS Head Fixation Frame is compatible with IMRIS surgical tables. **See Figure 1.**

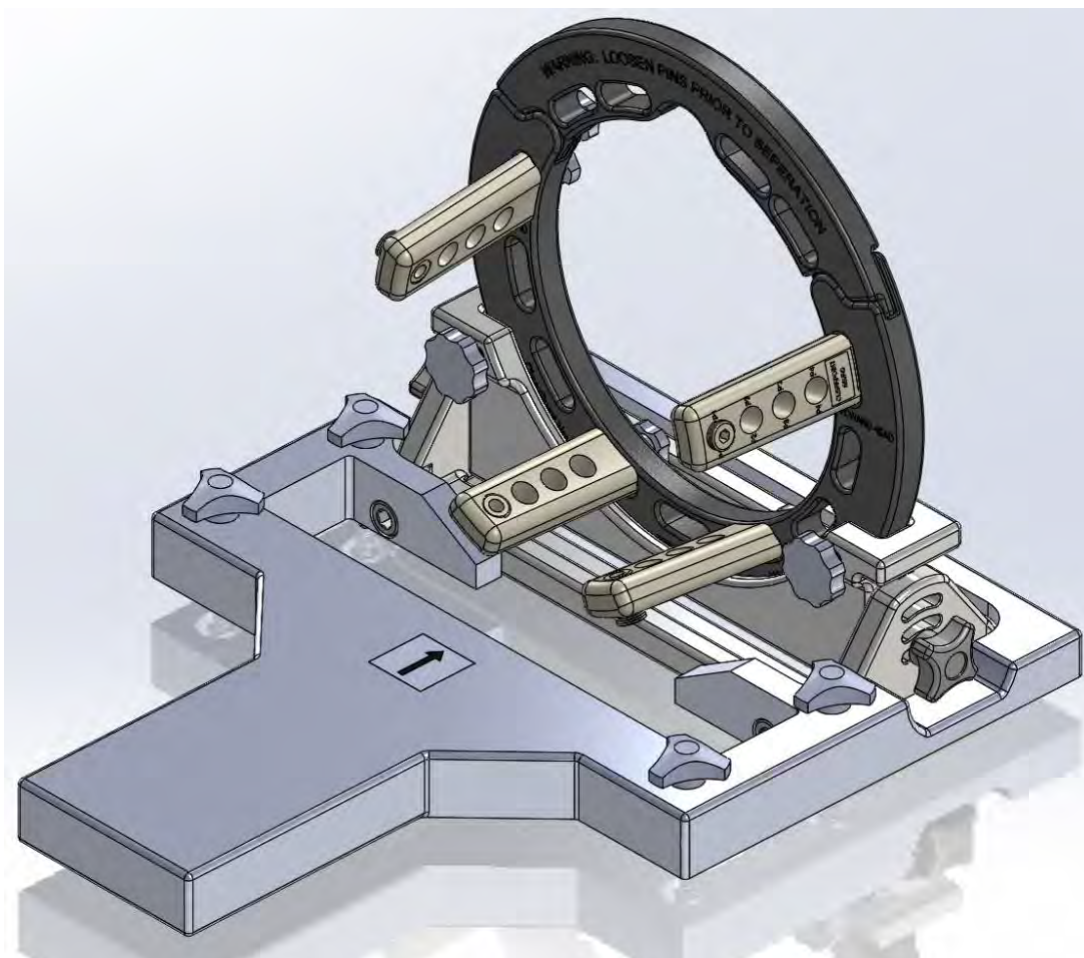**Figure 1: Inflexion Head Fixation Frame IMRIS Aera-Skyra**

**CP-ITE-01: ORT-200 IMRIS Table Extension (ORT 200 and 300 Series Tables ONLY)**

This Table Extension allows mounting the IMRIS Head Fixation Frame onto the IMRIS ORT 200 and ORT 300 Surgical Tables. See Figure 2.

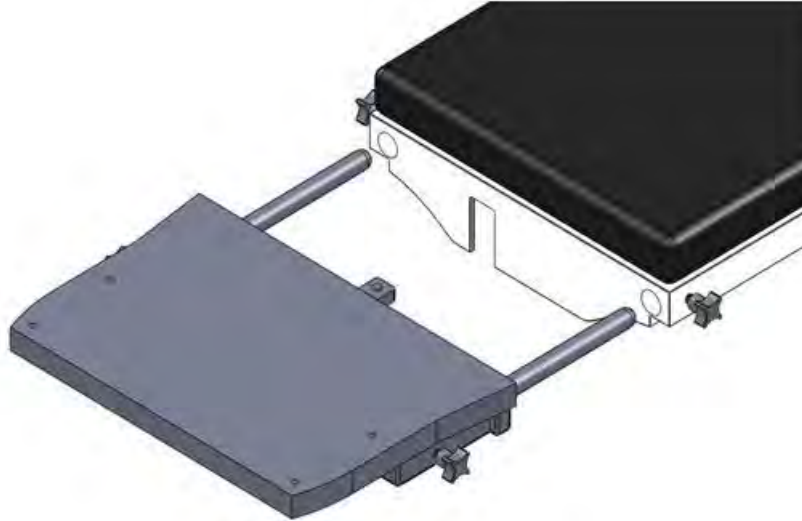
Figure 2: CP-ITE-01 Table Extension (left) with IMRIS ORT 200 Surgical Table (right)

**CP-ITE-02: ORT-400 IMRIS Table Extension (ORT 400 Series Tables ONLY)**

This Table Extension allows mounting the IMRIS Head Fixation Frame onto the IMRIS ORT 400 Surgical Table. See Figure 3.

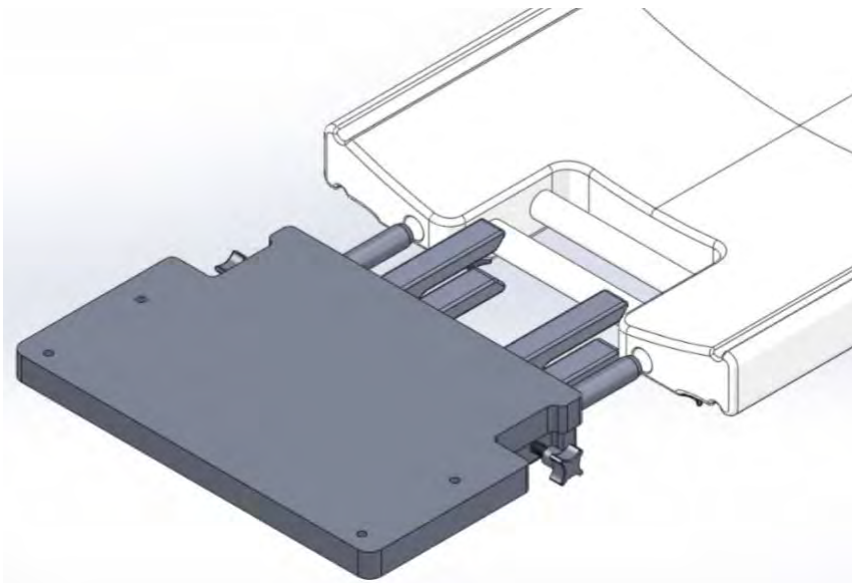
Figure 3: CP-ITE-02 Table Extension (left) with IMRIS ORT 400 Surgical Table (right)

**2. Use Instructions**

A ORT 100 – Installation onto the surgical table:

NOTE: The frame may be a different model.

1. The IMRIS End Table has 4 ea threaded holes on the top. See Figure 4.

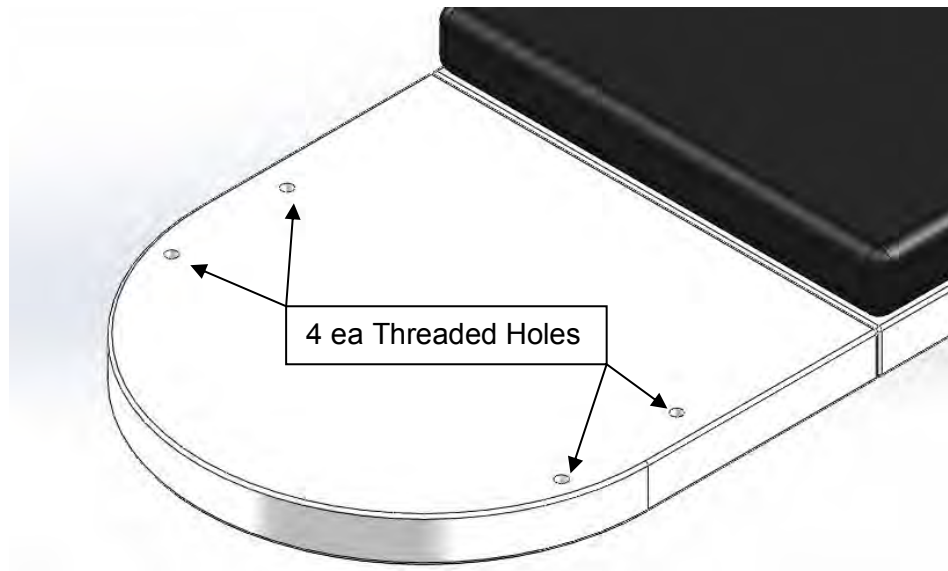
Figure 4.

2. Align the 4 ea. Thumbscrews on the Head Fixation Frame with the holes in the End Table. See Figure 5.

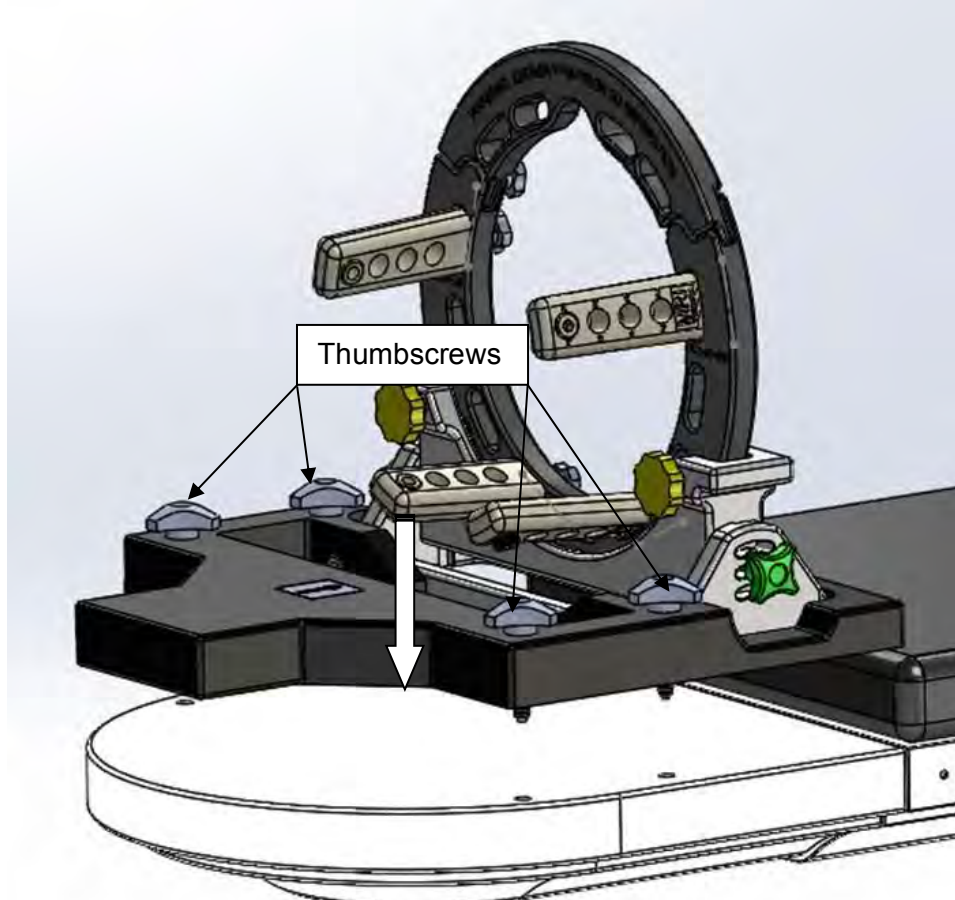
Figure 5.

3. Finger tighten the four (4) Table Base Screws to the IMRIS End Table. See Figure 6.

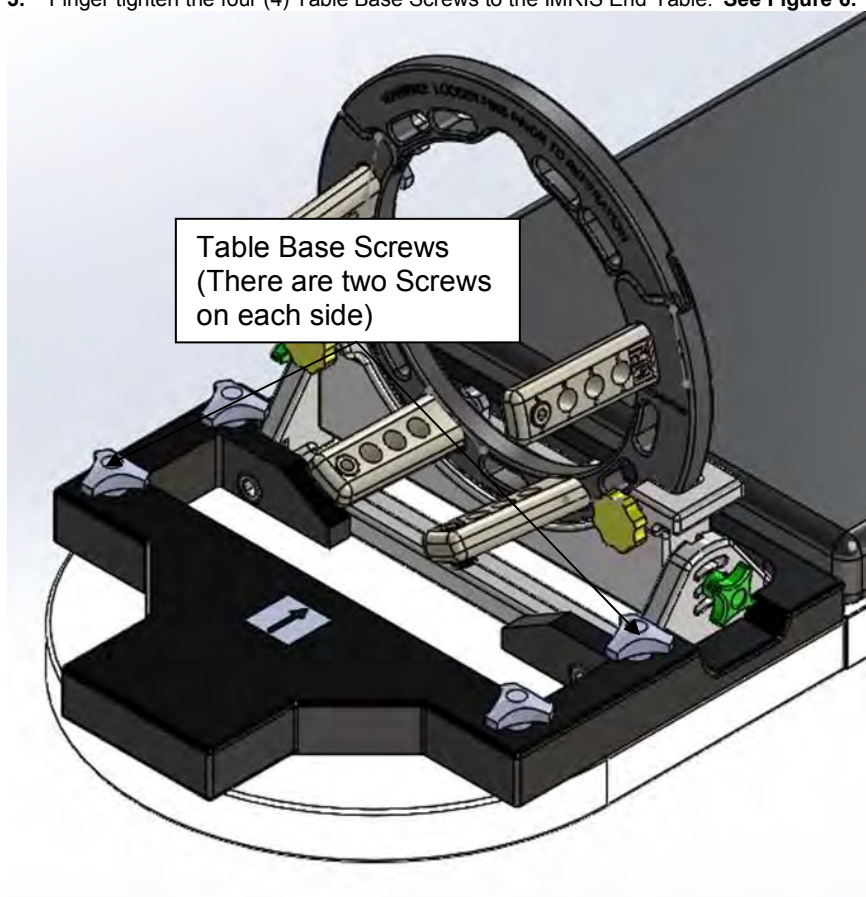
Figure 6.

**B. ORT 200 – Installation into the MRI Unit:**

1. Loosen the two outer Knobs, center Thumbscrew and two inner Thumbscrews on the Table Extension to aid in attachment to the Surgical Table. See Figure 7.

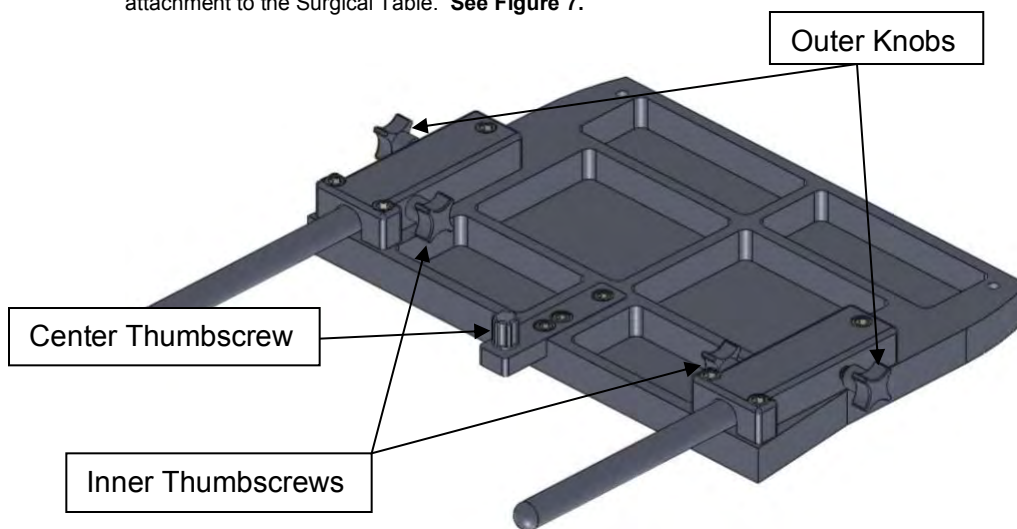
Figure 7.

2. Insert the two rods completely into the receiver in the surgical table. A side to side motion may be required during insertion. **See Figure 8.**

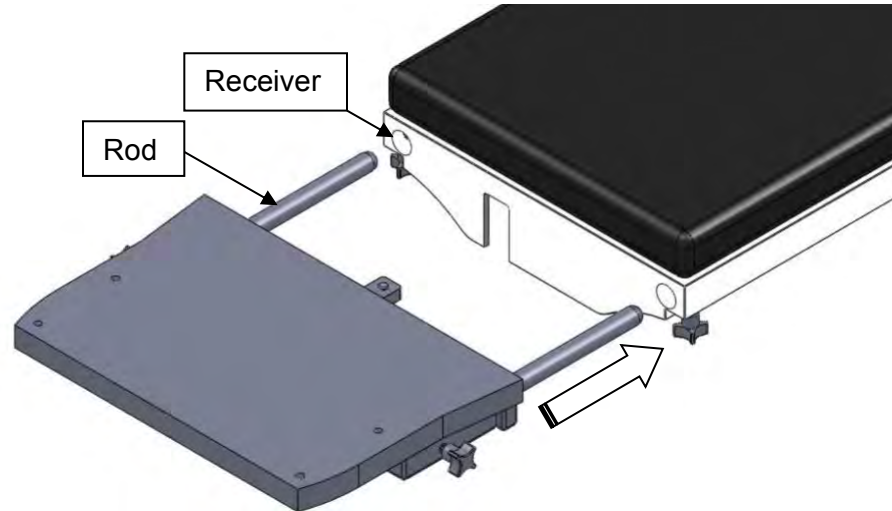
Figure 8.

3. Seat the Table Extension firmly against the surgical table. **See Figure 9.**

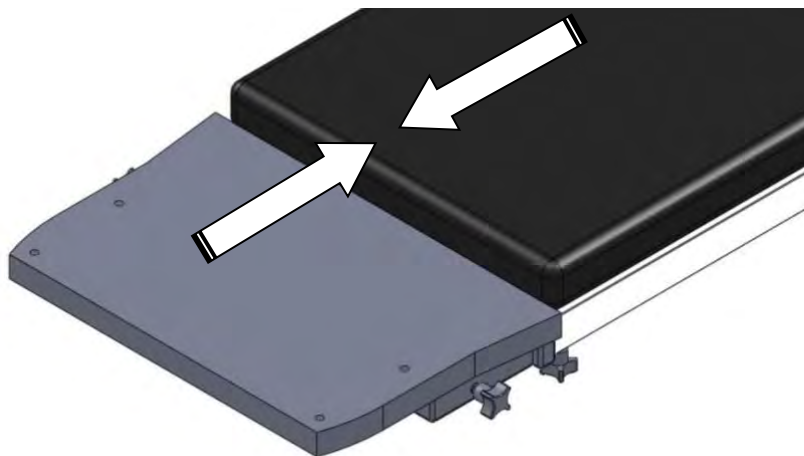
Figure 9.

4. Tighten the two inner Thumbscrews underneath the Table Extension. **See Figure 10.**

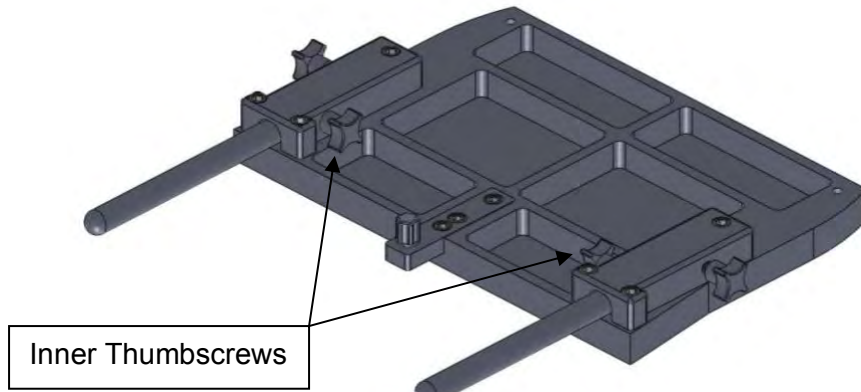
Figure 10.

5. Tighten the center Thumbscrew underneath the Table Extension. **See Figure 11.**

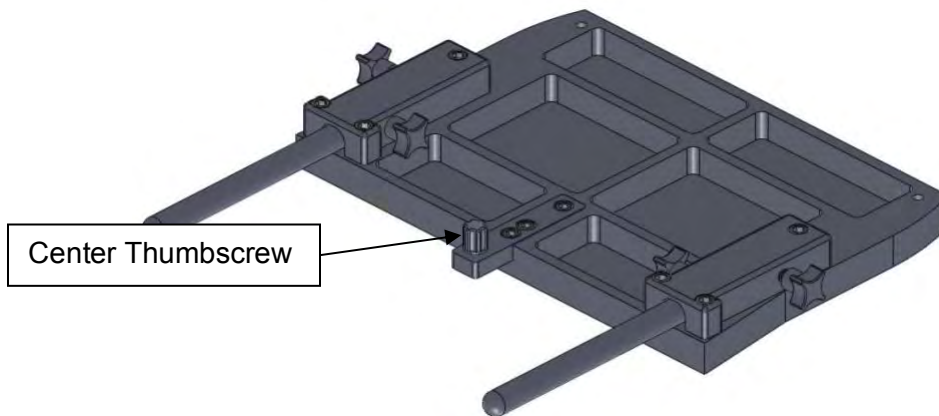
Figure 11.

6. Tighten the outer Knobs on the Table Extension. **See Figure 12.**

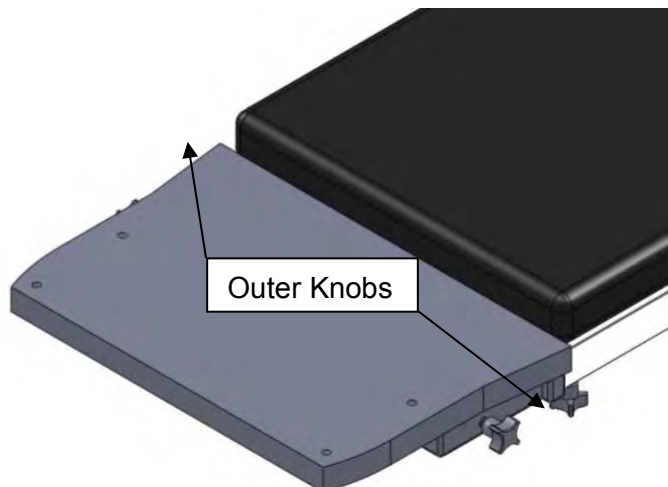
Figure 12.

7. Tighten the side Knobs on the Surgical Table to complete the securing of the Table Extension. **See Figure 13.**

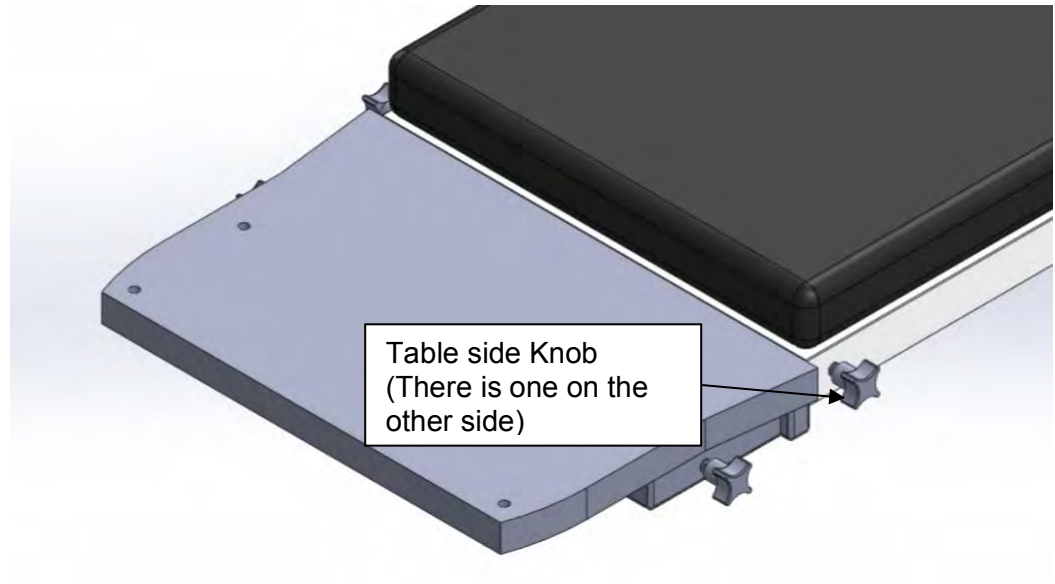
Figure 13.

8. Place the Head Fixation Frame assembly onto the ORT 200 Table. **See Figure 14.**

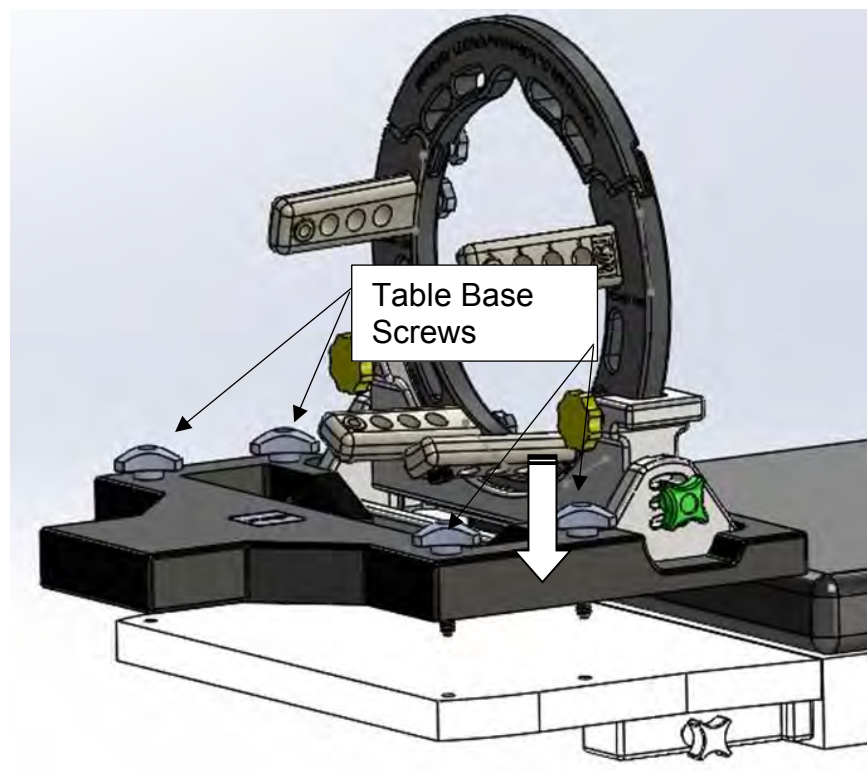
Figure 14.

9. Finger tighten the four (4) Table Base Screws to the ORT 200 Table. **See Figure 15.**

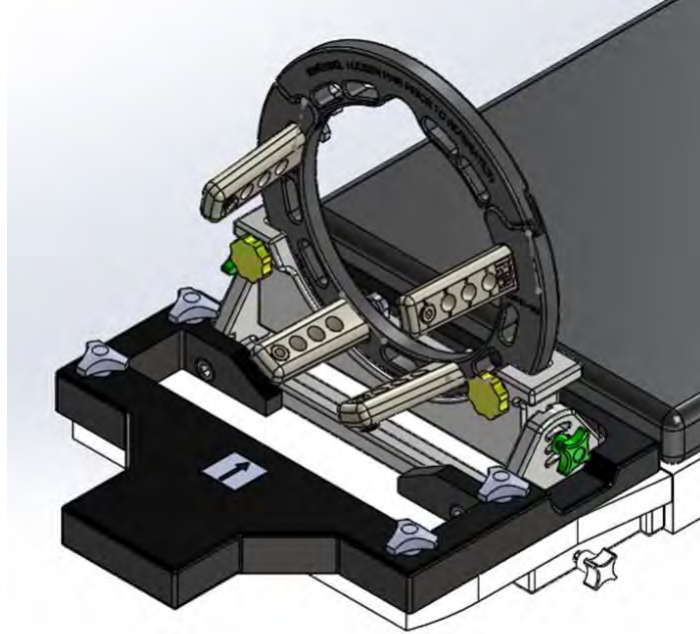
Figure 15.

**C. ORT 400 – Installation into the MRI Unit**

1. Loosen the two Thumb Screws on the Table Extension and the two Thumb Screws on the bottom of the Surgical Table to aid in attachment. **See Figure 16 and Figure 17.**

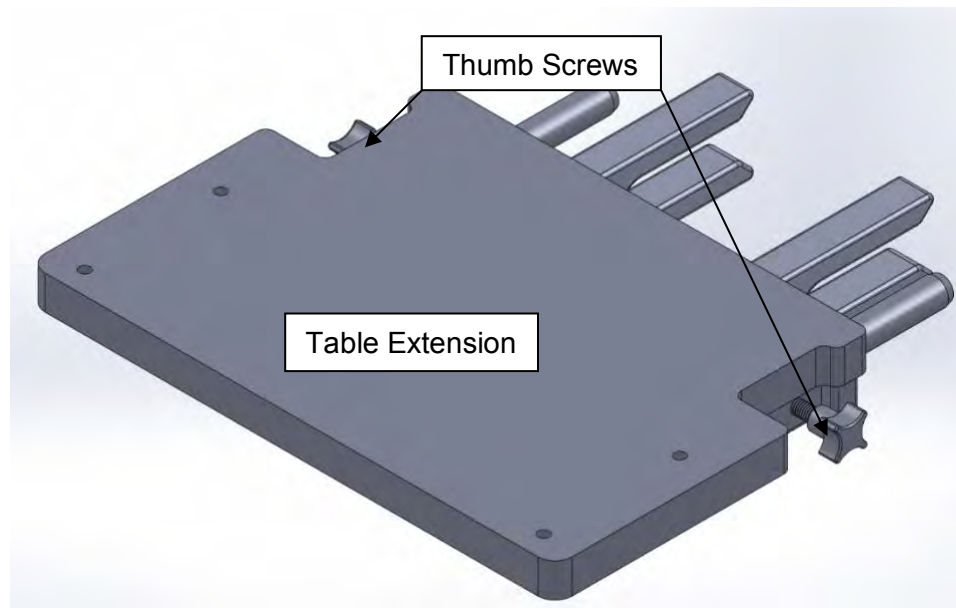
Figure 16.

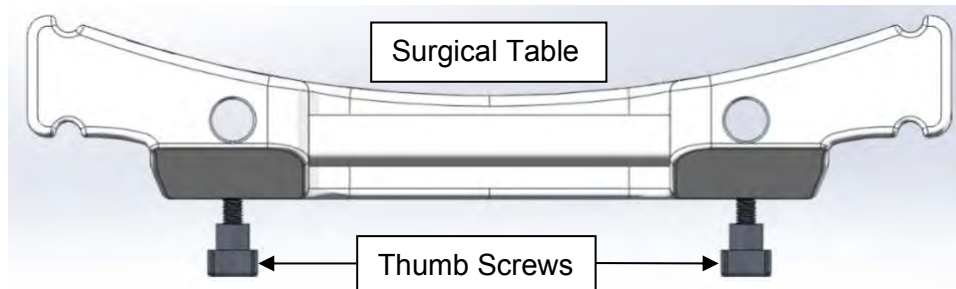
Figure 17.

2. Insert the two Rods completely into the receiver in the Surgical Table. A side-to-side motion may be required during insertion. **See Figure 18.**

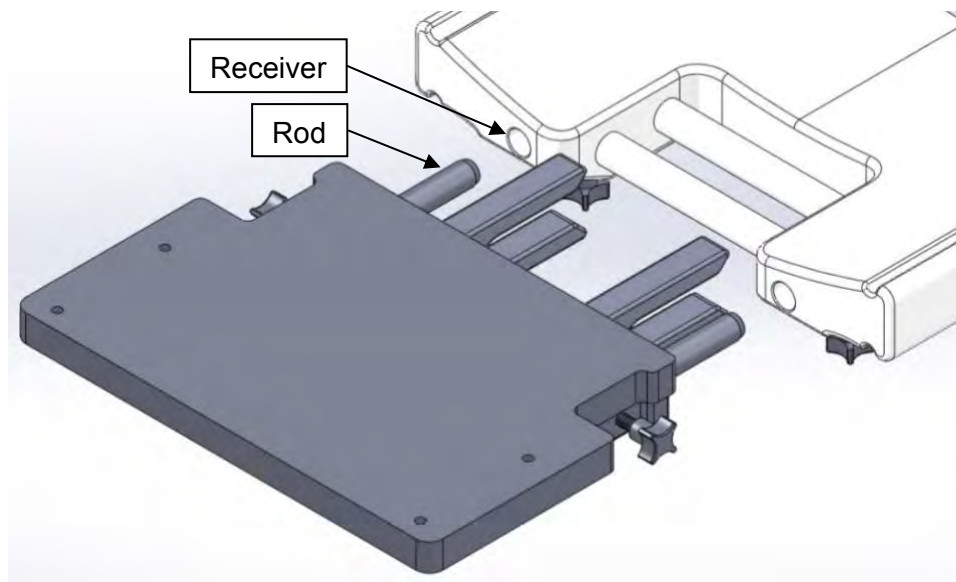
Figure 18.

3. Seat the Table Extension against the Surgical Table. **See Figure 19.**

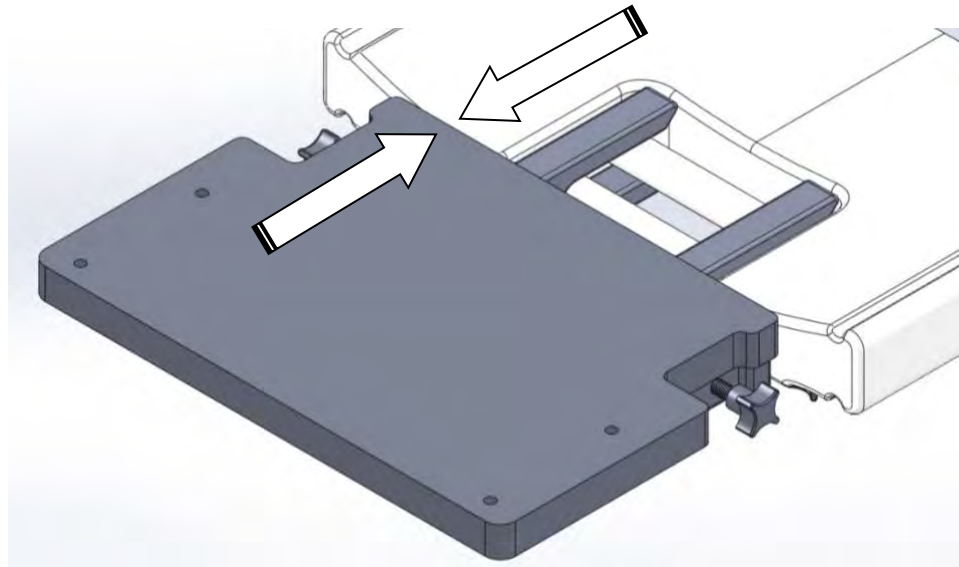
Figure 19.

4. Tighten the two Thumb Screws on the Table Extension. See Figure 20.

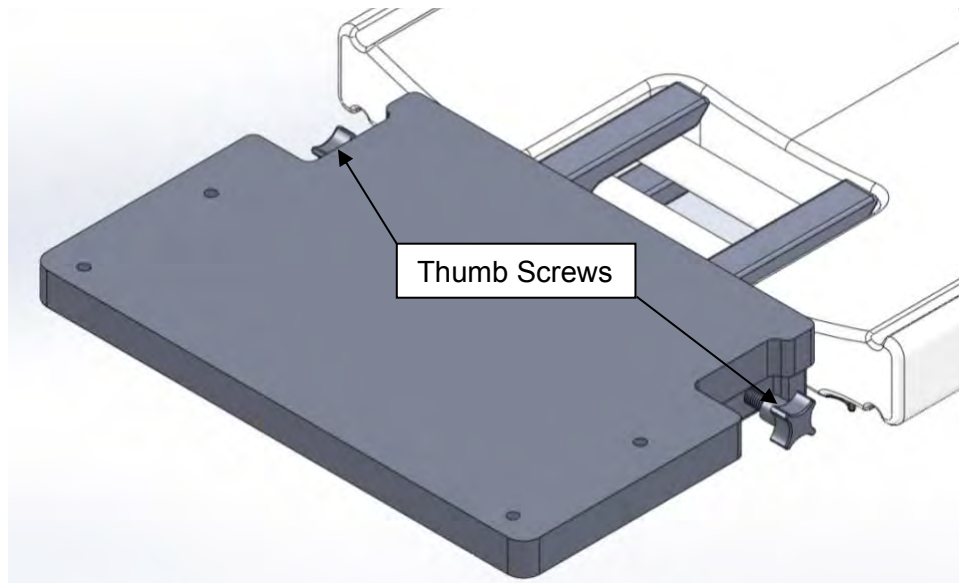
Figure 20.

5. Tighten the two Thumb Screws on the bottom of the Surgical Table to complete the securing of the Table Extension. See Figure 21.

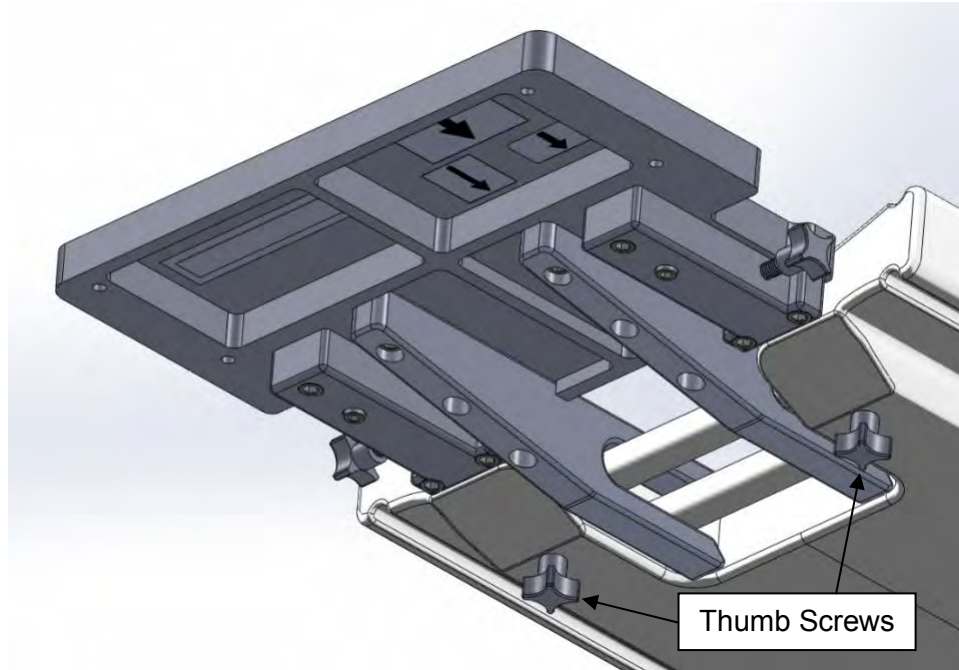
Figure 21.

6. Place the Inflexion Head Frame assembly onto the ORT 400 Table Extension. See Figure 22.

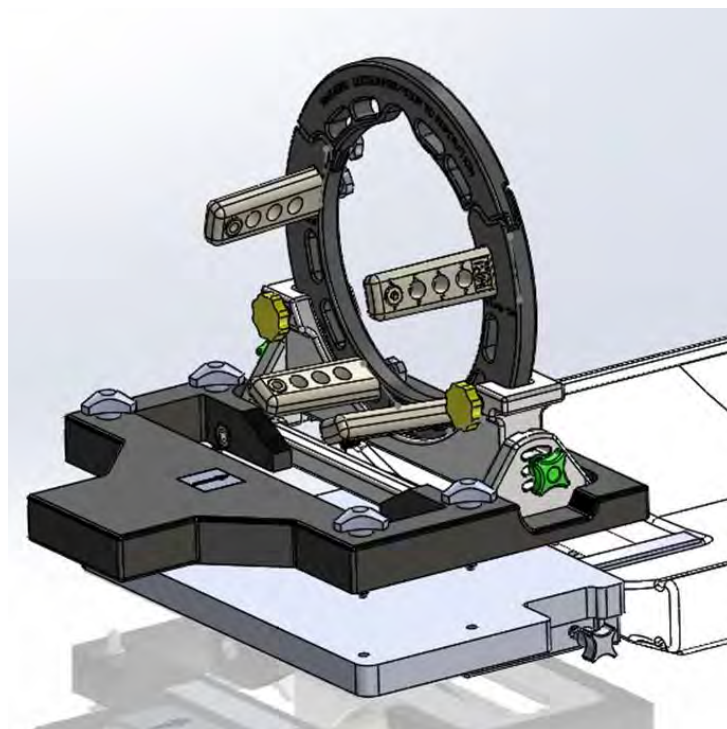
Figure 22.

7. Finger tighten the four (4) Frame Base Screws to the ORT 400 Table Extension. **See Figure 23.**

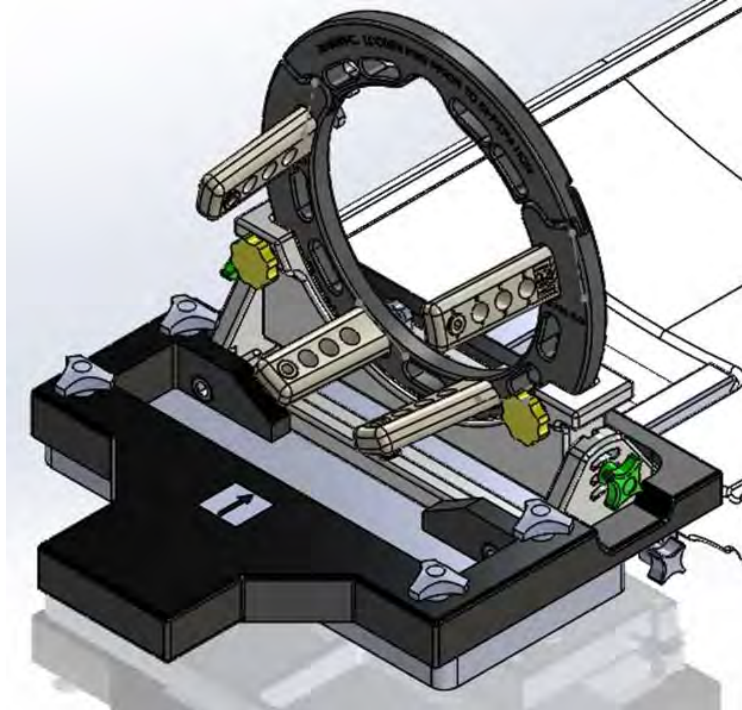
Figure 23.

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ClearPoint Neuro, Inc.