CLEARPOINT[®]

NEURO

Multi-Positional Head Fixation Frame and Table Mount, Philips Achieva

INSTRUCTIONS FOR USE - INSTALLATION

NOTE: These installation instructions are to be used in conjunction with the Universal Multi-Positional Head Fixation Frame Instructions for Use.

Table of Contents

- I. Device Description
- II. Use Instructions
 - A. Head Fixation Frame Installation

	<p>NOTE: These installation instructions are to be used in conjunction with the Universal Multi-Positional Head Fixation Frame Instructions for Use.</p>
---	---

I. Device Description

Package Contents:

**CP-MPHFF-412-01: CLEARPOINT® Multi-Positional Head Fixation Frame
(Philips Achieva - MR Table Compatible)**

Head Fixation Frame, Upper Fixation Screws (4), Lower Fixation Screws (4), Drive Wheels (2)

Associated Devices:

CP-FFSMPU-01: Upper Fixation Frame Screws (replacements in case of damage or loss)

Upper Fixation Screws (4)

CP-FFSMPL-01: Lower Fixation Frame Screws (replacements in case of damage or loss)

Lower Fixation Screws (4)

NGS -SP-01: Skull Pin Tips

MR Compatible Skull Pin Tips

NGS-SP-02: Pediatric Skull Pin Tips


MR Compatible Skull Pin Tips, Pediatric

The Multi-Positional Head Fixation Frame is designed to immobilize the patient's head during the surgical procedure. The patient's head is rigidly attached to the Head Fixation Frame by the physician in the desired position (lateral, oblique, prone or supine) for the planned surgical procedure.

CP-MPHFF-412-01 (Philips Achieva - MR Table Compatible)

The Multi-Positional Head Fixation Frame is compatible with the following MRI Systems:

- Philips Achieva 1.5T
- Philips Intera 1.5T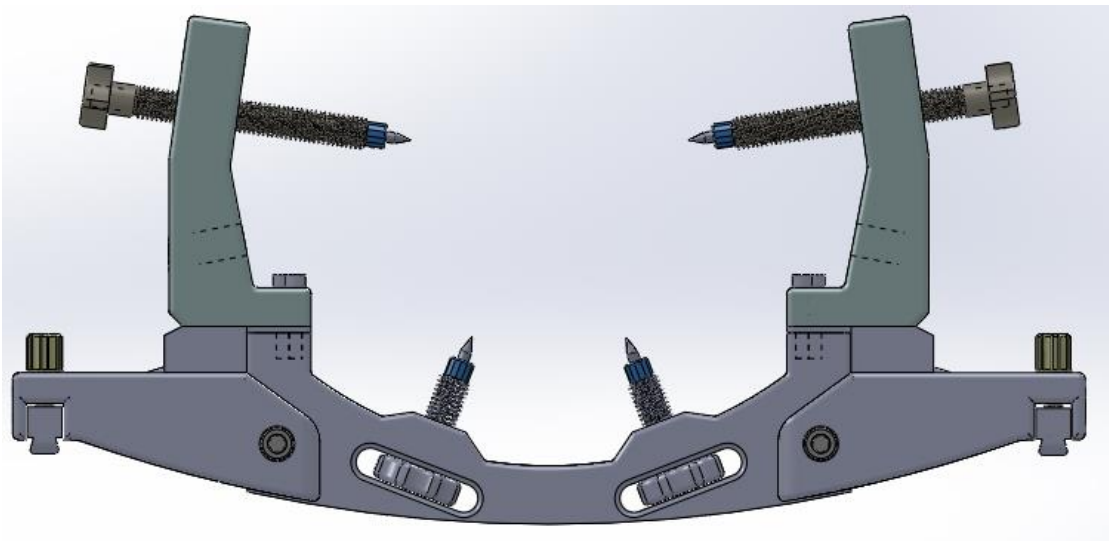


Head Fixation Frame


II. Use Instructions

A. Head Fixation Frame Installation


1. The Cradle has a groove on each side running full length.
2. At the superior end of the Cradle there are two cut-outs, one on each side.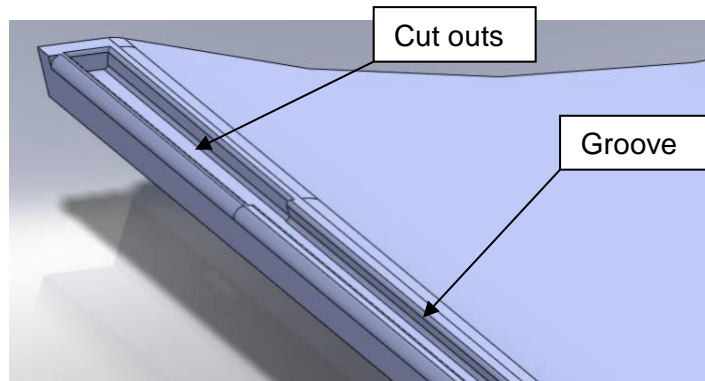


3. Loosen the four (4) Thumb Screws of the Head Fixation Frame.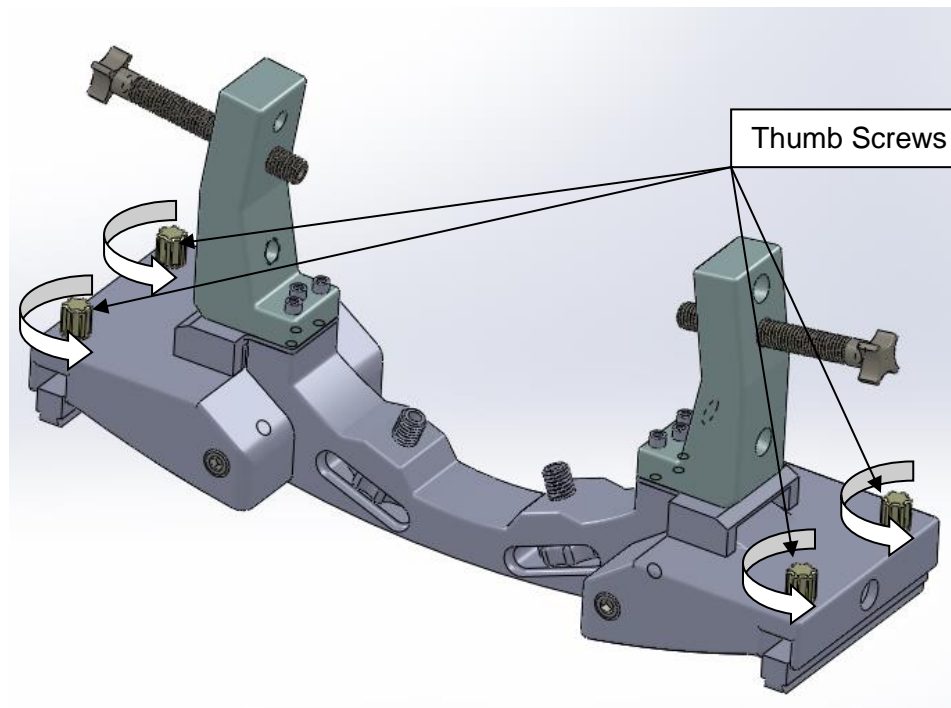

4. Install the Head Fixation Frame onto the MRI Table, by setting the Dovetails into the cut-outs at the end of the table.

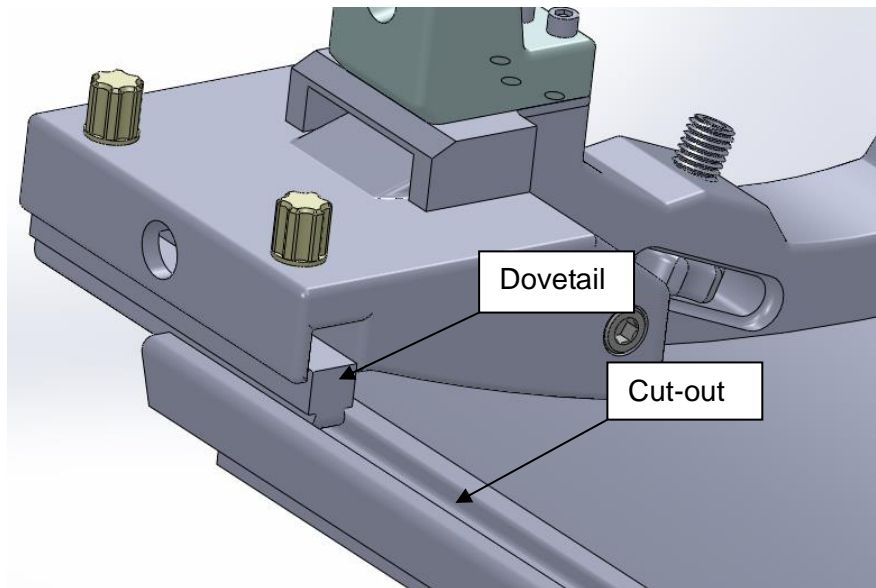
5. Gently slide the Head Fixation Frame to the desired position.

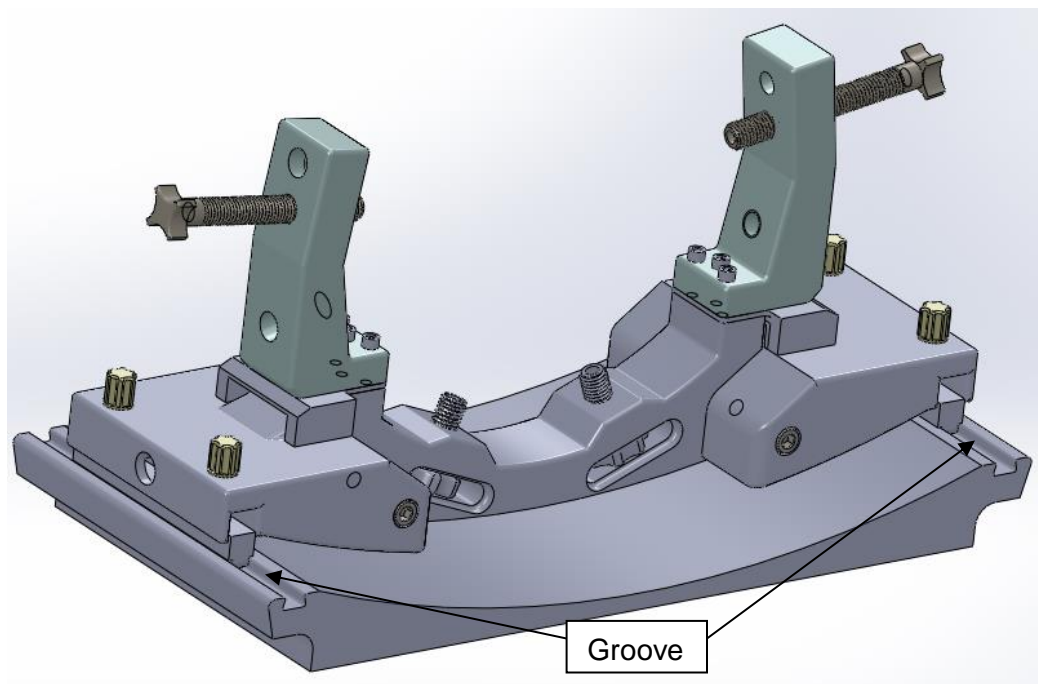6. Gently tighten the Thumb Screws, to secure the Fixation Frame to the Table.

Note: It is very important not to over tighten the Thumb Screws, over tightening can damage the MRI table

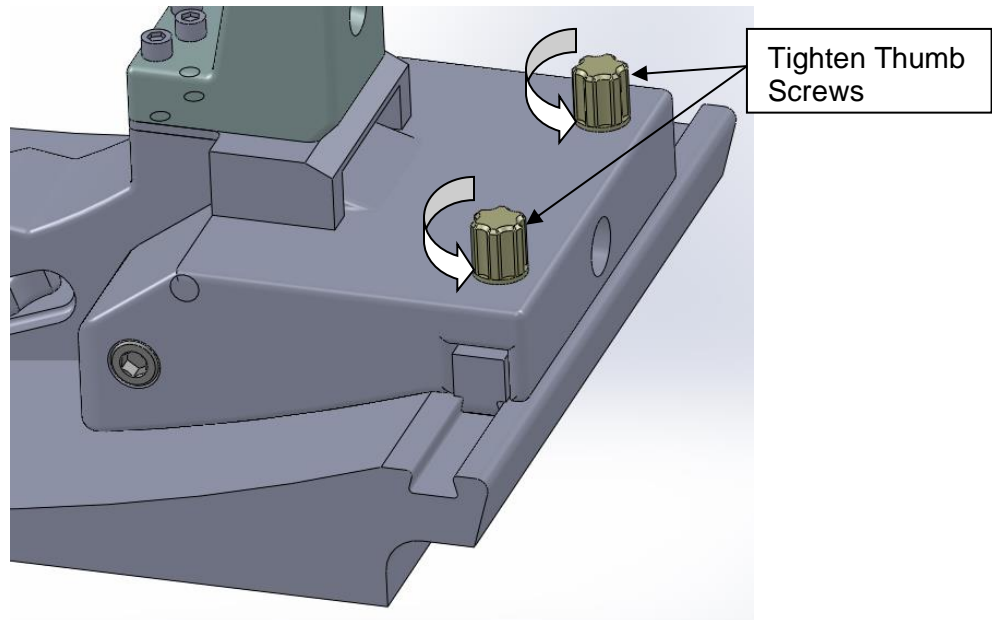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ClearPoint Neuro, Inc.