# CLEARPOINT<sup>®</sup>

## NEURO

**Philips - Ingenia Multi-Positional Head Fixation Frame**

**INSTRUCTIONS FOR USE - INSTALLATION**

**NOTE:** These installation instructions are to be used in conjunction with the Universal Multi-Positional Head Fixation Frame Instructions for Use.

**CE**

## Table of Contents

- 1. **Device Description**
- 2. **Use Instructions**
  - A. **Head Fixation Frame Installation**

	<p><b>NOTE:</b> These installation instructions are to be used in conjunction with the Universal Multi-Positional Head Fixation Frame Instructions for Use.</p>
---	---

### 1. **Device Description**

#### **Package Contents:**

**CP- MPHFF-422-01: ClearPoint Multi-Positional Head Fixation Frame (New Build)**  
 (Philips Ingenia - MR Table Compatible)

Head Fixation Frame, Upper Fixation Screws (4), Lower Fixation Screws (4), Drive Wheels (2)

**-OR-**

**CP- MPFA-422-01: ClearPoint Multi-Positional Head Fixation Frame (Upgraded)**  
 (Philips Ingenia - MR Table Compatible)

Head Fixation Frame, Upper Fixation Screws (4), Lower Fixation Screws (4), Drive Wheels (2)

#### **Associated Devices:**

**CP-FFSMPU-01: Upper Fixation Frame Screws (replacements in case of damage)**  
 Upper Fixation Screws (4)

**CP-FFSMPL-02: Fixation Frame Screws for Philips (replacements in case of damage)**  
 Lower Fixation Screws (4)


**NGS-SP-01: Skull Pin Tips**  
 MR Compatible Skull Pin Tips

**NGS-SP-02: Pediatric Skull Pin Tips**  
 MR Compatible Skull Pin Tips, Pediatric


The Multi-Positional Head Fixation Frame is designed to immobilize the patient's head during the surgical procedure. The patient's head is rigidly attached to the Head Fixation Frame by the physician in the desired position (lateral, oblique, prone or supine) for the planned surgical procedure.

The Philips Ingenia Multi-Positional Head Fixation Frame is compatible with the following MRI Systems:

- Philips Ingenia 1.5T and 3T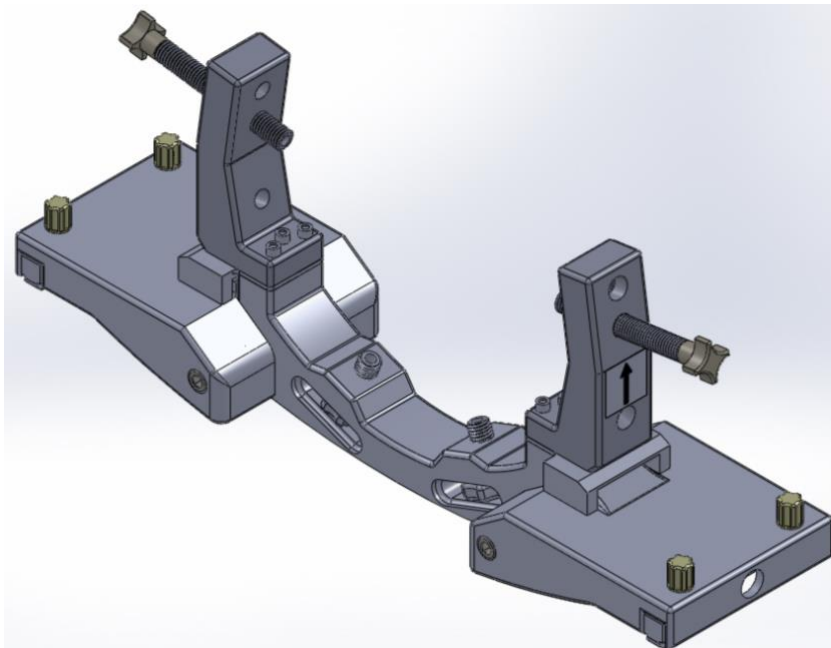


## 2. Use Instructions

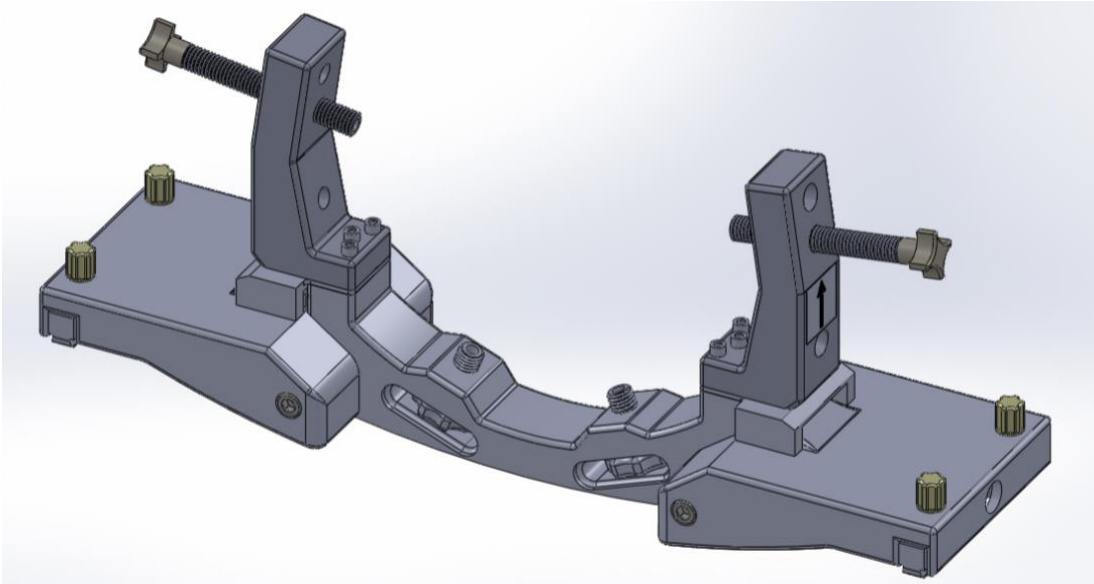
### A. Head Fixation Frame Installation


1. The Cradle has two grooves on each side running the full length of the Cradle.
2. At the superior end of the Cradle there is a cut out on the outer groove only, one on each side. Note: The Philips Ingenia Head Fixation Frame is designed to fit the outer groove with the cut outs.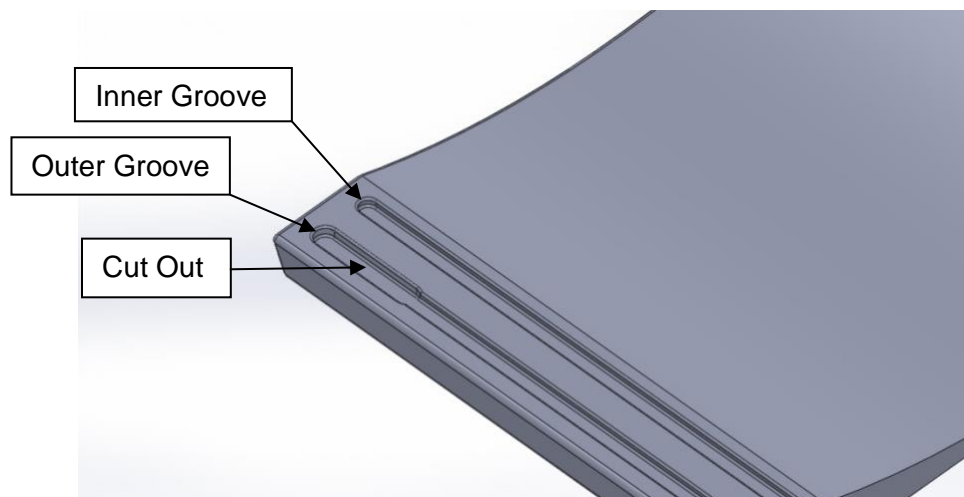

3. Loosen the four (4) Thumb Screws of the Head Fixation Frame.

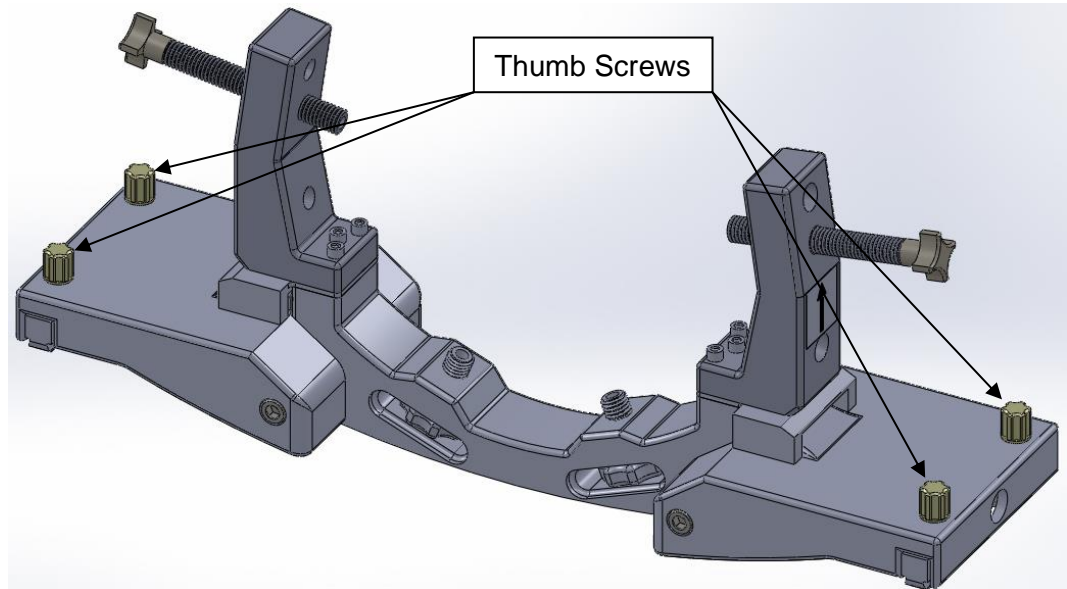
4. Install the Head Fixation Frame onto the MRI Table by inserting the Dovetails into the cut-outs at the end of the table.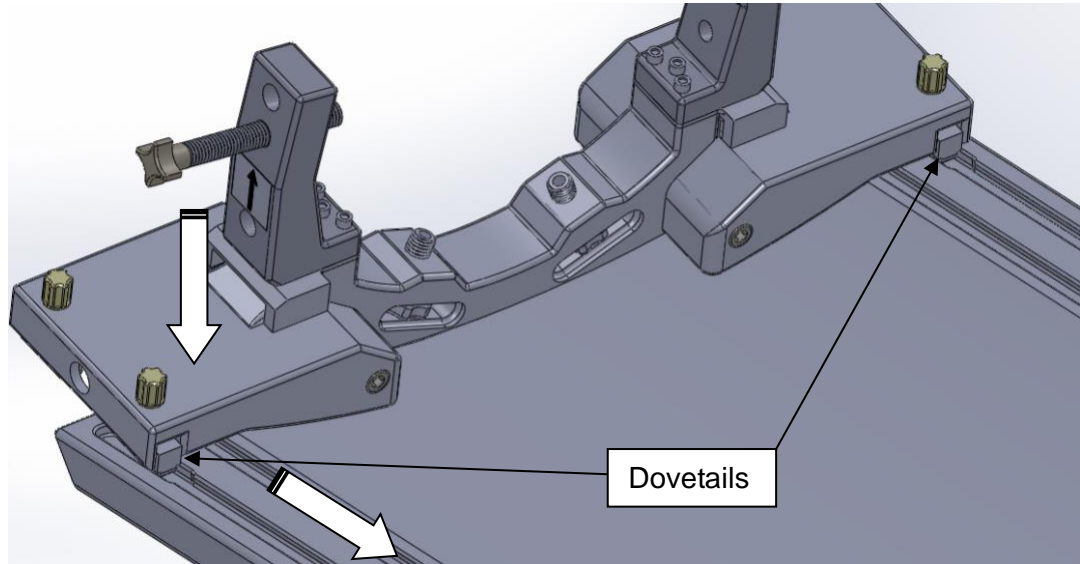

5. Slide the Head Fixation Frame forward enough that all four Dovetails can be fully engaged. Gently tighten all four of the Thumb Screws to secure the Head Fixation Frame to the Table.

**Caution:** It is very important not to over tighten the Thumb Screws, over tightening can damage the Head Fixation Frame and/or the MRI Table.

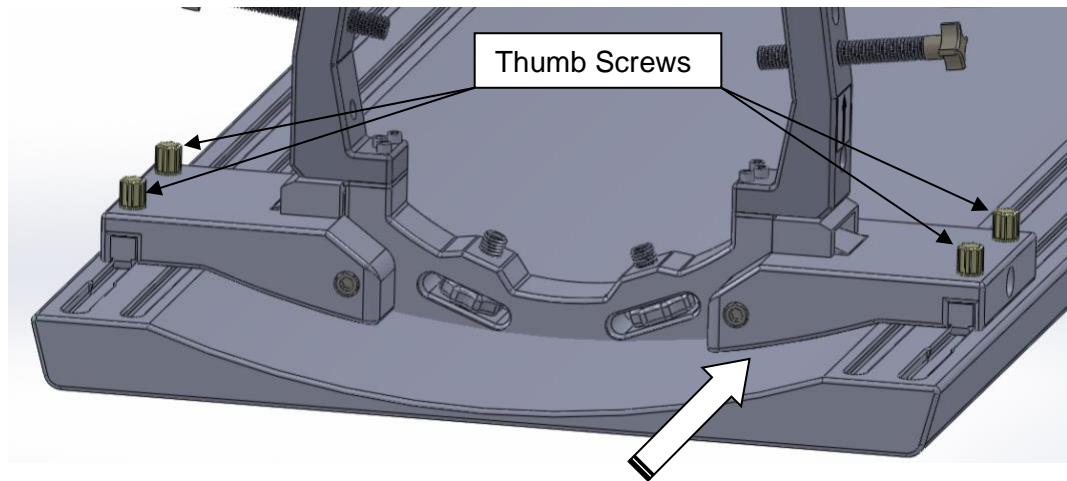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ClearPoint Neuro, Inc.