CLEARPOINT[®]

NEURO

SMS Avanto/Espree/Symphony/Sonata Multi-Positional Head Fixation Frame

INSTRUCTIONS FOR USE - INSTALLATION


NOTE: These installation instructions are to be used in conjunction with the Universal Multi-Positional Head Fixation Frame Instructions for Use.

Table of Contents

1. **Device Description**
2. **Use Instructions**
 - A Head Fixation Frame Installation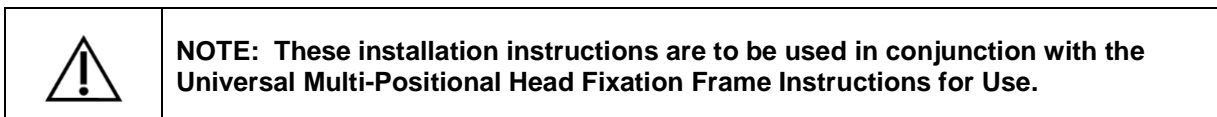

1. Device Description

Package Contents:

CP-MPHFF-112-01: SMS Avanto/Espree/Symphony/Sonata Table Head Fixation Frame

(SMS Avanto/Espree/Symphony/Sonata/Trio/Verio - MR Table Compatible)

Head Fixation Frame, Upper Fixation Screws (4), Lower Fixation Screws (4), Drive Wheels (2), Thumb Screws (2)

Associated Devices:

CP-FFSMPU-01: Upper Fixation Frame Screws (replacements in case of damage)

Upper Fixation Screws (4)

CP-FFSMPL-01: Fixation Frame Screws (replacements in case of damage)

Lower Fixation Screws (4)

NGS-SP-01: Skull Pin Tips

MR Compatible Skull Pin Tips

NGS-SP-02: Pediatric Skull Pin Tips

MR Compatible Skull Pin Tips, Pediatric

The Multi-Positional Head Fixation Frame is designed to immobilize the patient's head during the surgical procedure. The patient's head is rigidly attached to the Head Fixation Frame by the physician in the desired position (lateral, oblique, prone or supine) for the planned surgical procedure.

This Multi-Positional Head Fixation Frame is compatible with the following MRI Systems:

- Siemens 1.5T and 3T

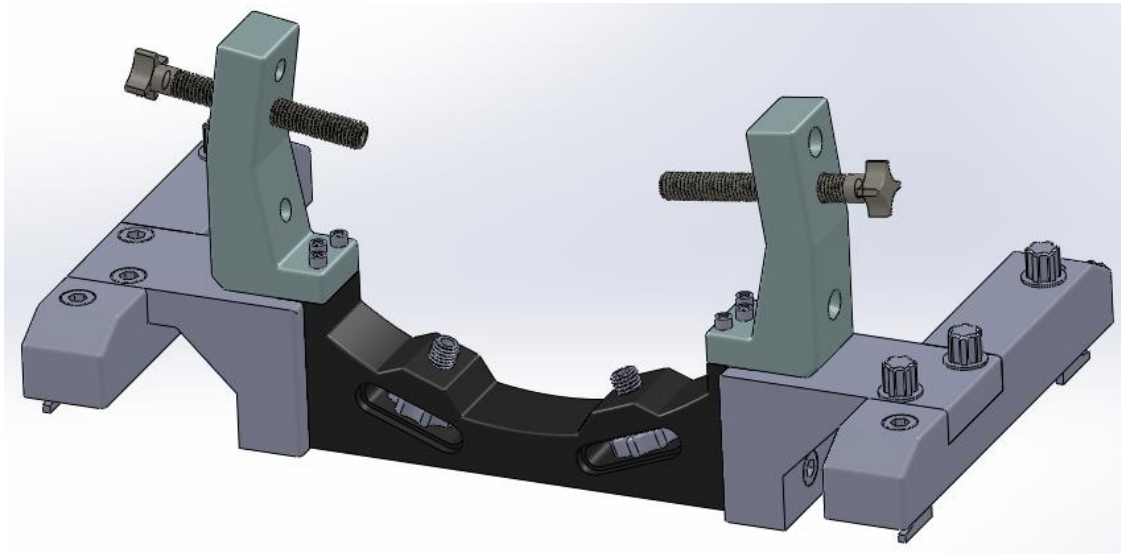
Figure 1: Head Fixation Frame with Table Mounts

A. **Head Fixation Frame Installation**

1. The scanner table has slotted holes equally spaced on both side of the table. **See Figure 2.**

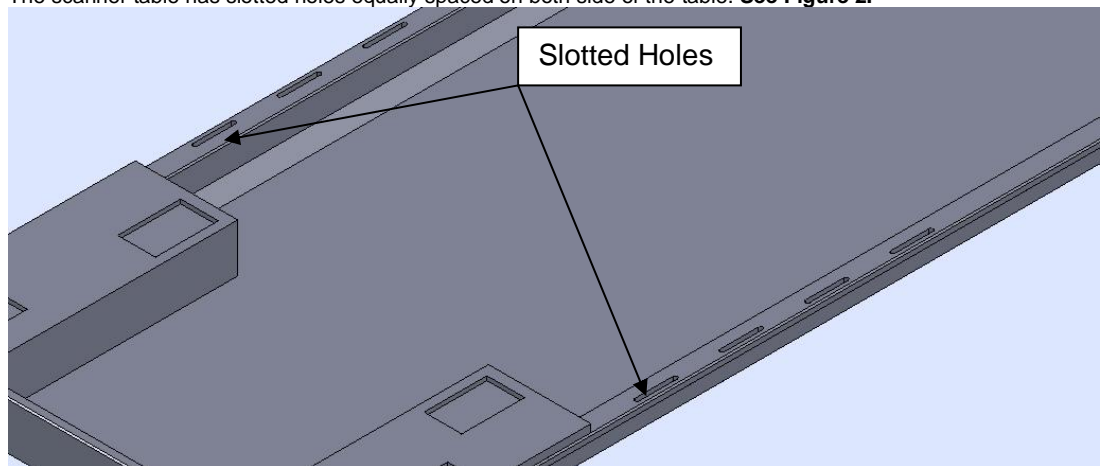
Figure: 2: Scanner Table

2. Loosen the two width adjustment Thumbscrews and adjust the width of the Head Frame so the Fixed Toe Inserts fit into the slotted holes on both sides of the table. **See Figures 3a and 3b.**

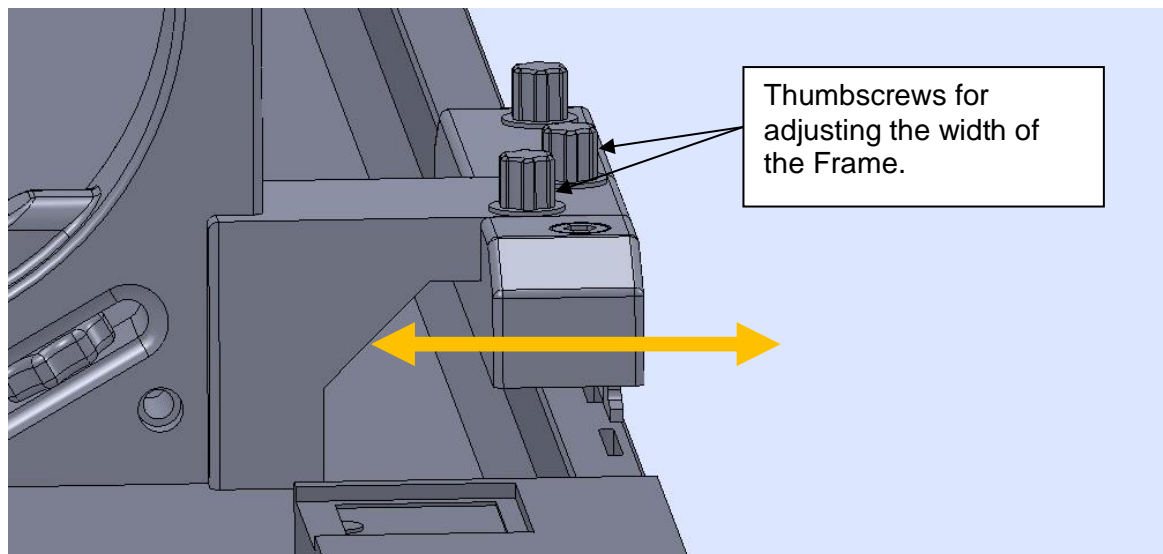
Figure: 3a: Width Adjustment

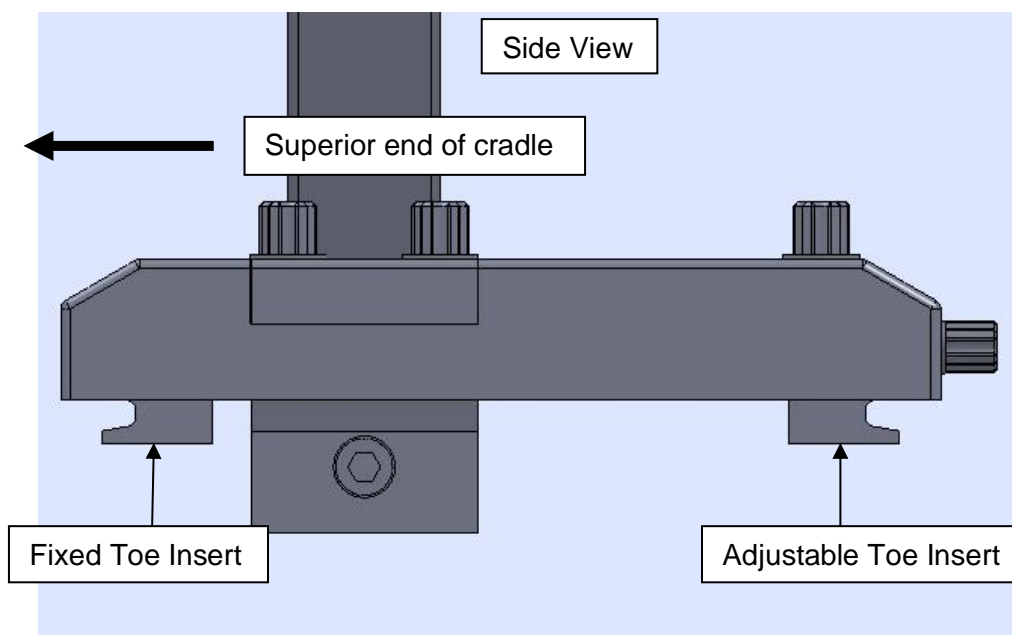
Figure: 3b: Frame Orientation on Table

3. Loosen the Top Thumbscrews, then turn the Front Thumbscrews counterclockwise until the Adjustable Toe Inserts (both sides) are all the way back. **See Figure 4.**

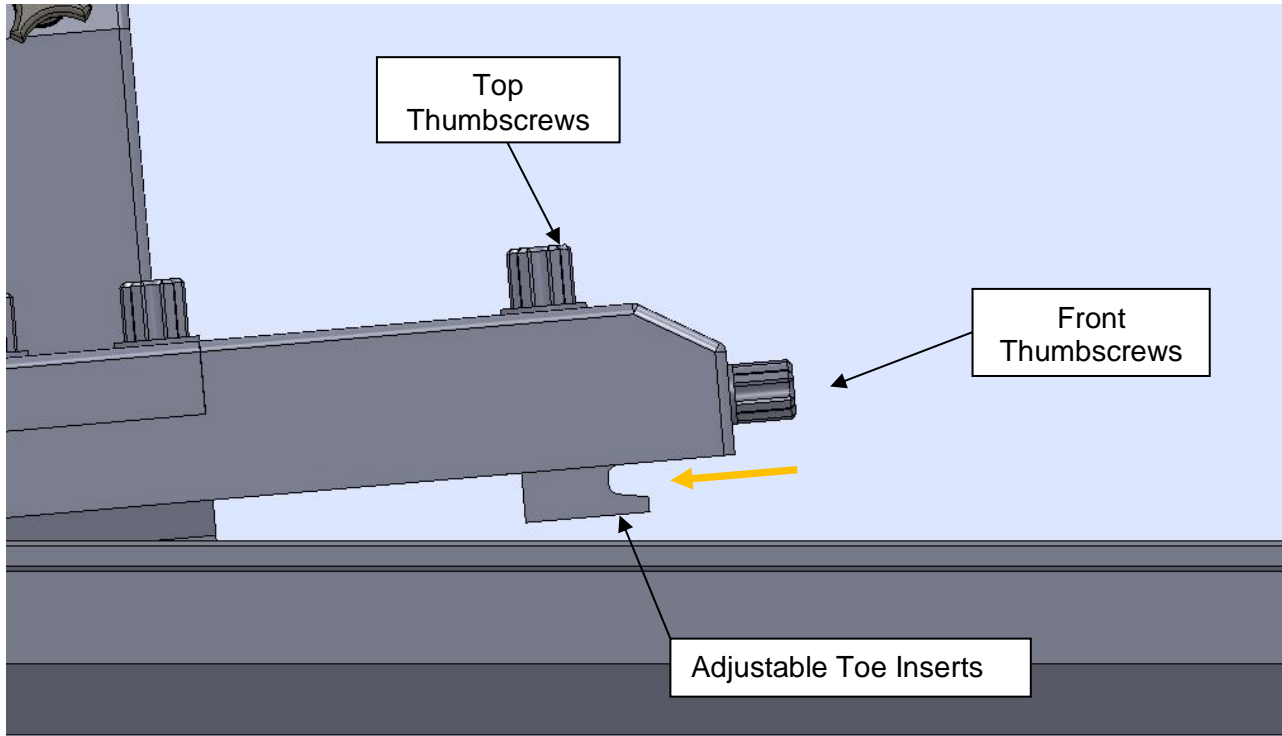
Figure 4: Adjusting Toe Inserts

4. Insert the two Fixed Toe Inserts into the slotted holes and then insert the two Adjustable Toe Inserts. **See Figure 5.**
Note: If the Toe Inserts will not recede far enough to fit into the Table, remove the front Thumb Screws and use the longer Thumb Screws provided.

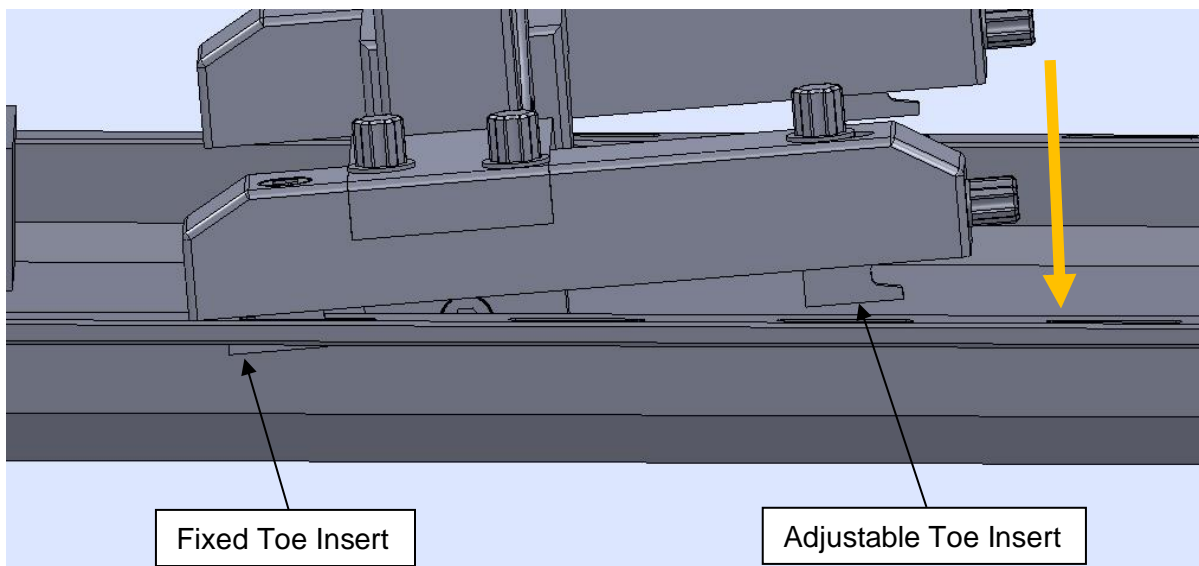
Figure 5: Insert Frame into slots on the Table.

5. Tighten the front two Thumbscrews until the Adjustable Toe Inserts are snug and then tighten the top two Thumbscrews to lock the Toe inserts into place. Finally tighten the two width adjustment Thumbscrews to secure the Head Frame to the table. See Figure 6.

CAUTION: Over tightening the front two Thumbscrews may result in breaking the Toe Inserts.

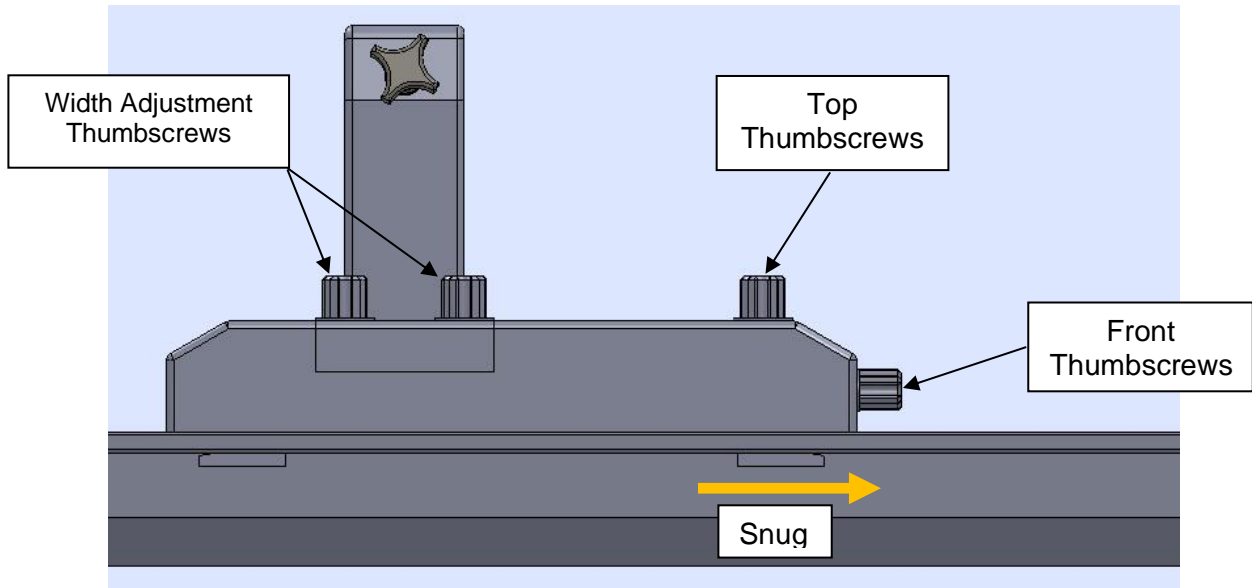
Figure 6: Tightening Frame onto Table.

6. See the Universal Multi-Positional Head Fixation Frame Instructions for Use after installation of the Head Fixation Frame.

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ClearPoint Neuro, Inc.