CLEARPOINT[®]

NEURO

SMS – Sola/Vida Multi-Positional Head Fixation Frame

INSTRUCTIONS FOR USE- INSTALLATION

NOTE: These installation instructions are to be used in conjunction with the Universal Multi-Positional Head Fixation Frame Instructions for Use.

Table of Contents

1. Device Description

2. Use Instructions

A. Head Fixation Frame Installation

	<p>NOTE: These installation instructions are to be used in conjunction with the Universal Multi-Positional Head Fixation Frame Instructions for Use.</p>
---	---

1. Device Description

Package Contents:

CP-MPHFF-132-01: ClearPoint Multi-Positional Head Fixation Frame

(SMS – Sola/Vida – MR Table Compatible) Head Fixation Frame, Upper Fixation Screws (6), Lower Fixation Screws (4) Drive Wheels (2)

Associated Devices:

CP-FFSMPU-01: Upper Fixation Frame Screws (replacements in case of damage)

Upper Fixation Screws (6)

CP-FFSMPL-01: Fixation Frame Screws for GE, Siemens and IMRIS (replacements in case of damage)

Lower Fixation Screws (4)

CP-FFSMPL-02: Fixation Frame Screws for Philips (replacements in case of damage)

Lower Fixation Screws (4)

NGS-SP-01: Skull Pin Tips

MR Compatible Skull Pin Tips

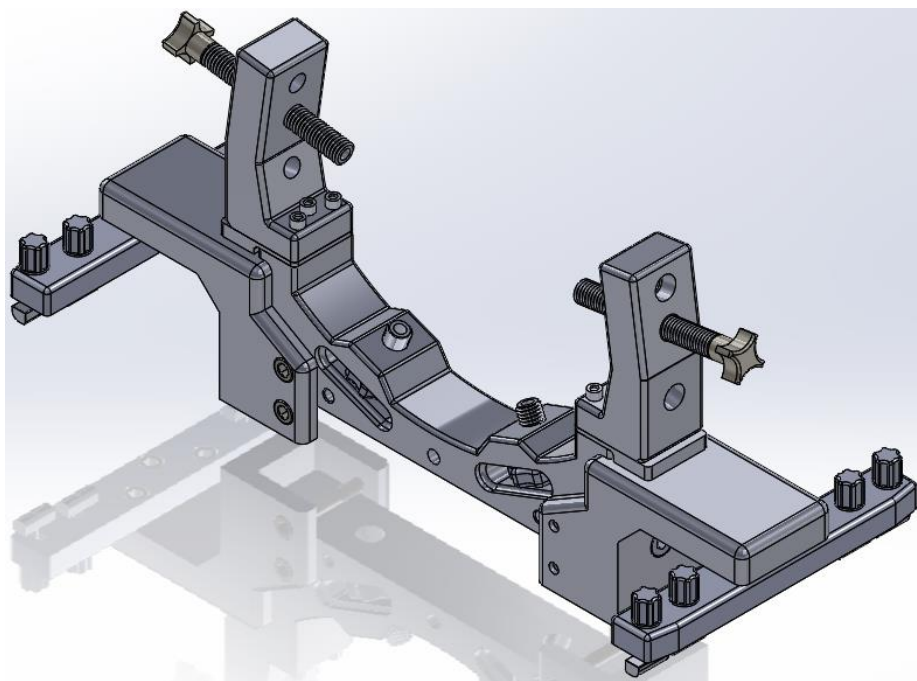
NGS-SP-02: Skull Pin Tips

MR Compatible Skull Pin Tips, Pediatric

The Multi-Positional Head Fixation Frame is designed to immobilize the patient’s head during the surgical procedure. The patient’s head is rigidly attached to the Head Fixation Frame by the physician in the desired position (lateral, oblique, prone or supine) for the planned surgical procedure.

The IMRIS Multi-Positional Head Fixation Frame is compatible with the following MRI Systems:


- SMS – Sola/Vida 1.5T and 3T


2. Use Instructions

A. Head Fixation Frame Installation

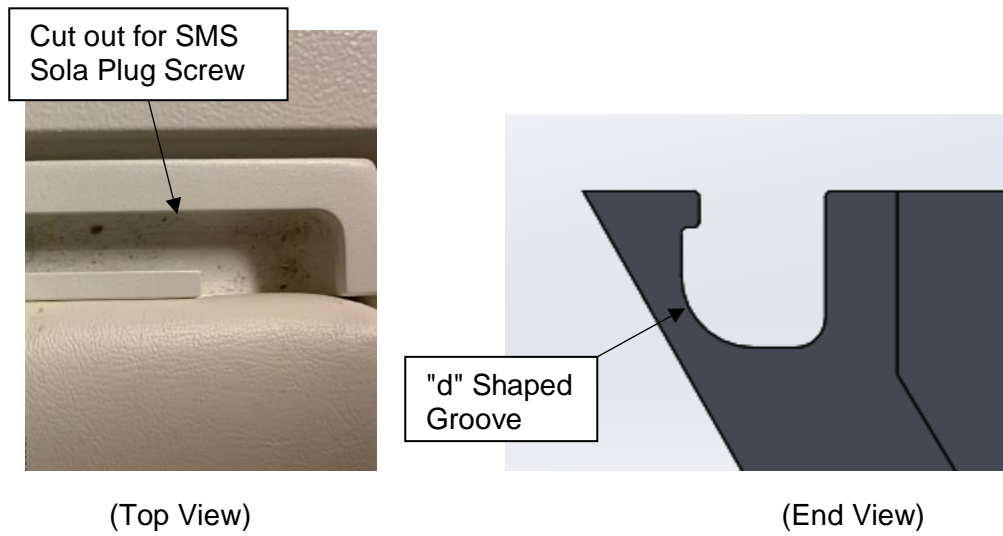
1. The cradle has a “d” shaped groove on each side as seen in the top view in Figure 2a and end view in the right figure below. There are also cut outs seen in the left figure below that the SMS Sola Plug Screw will fit in.

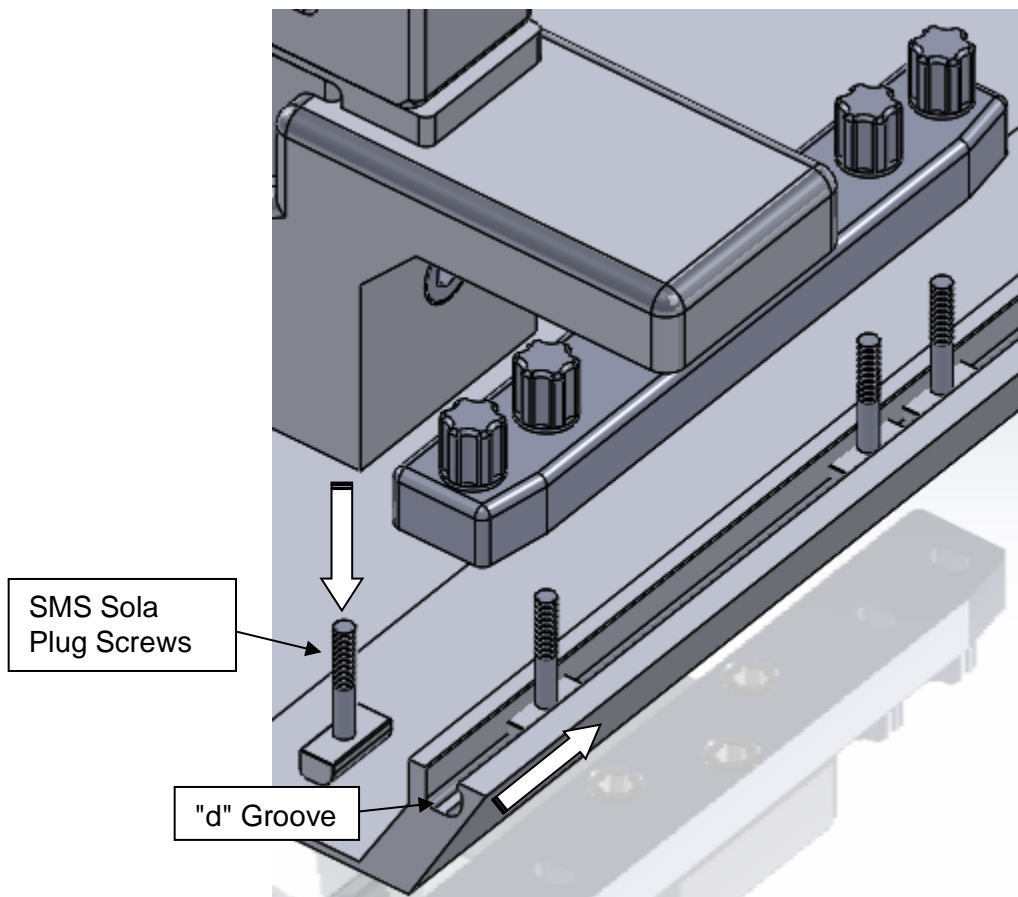2. Remove the 8ea SMS Sola Plug Screw and place them down into the "d" groove aligning them with the 2ea Table Mounts (one on each side).

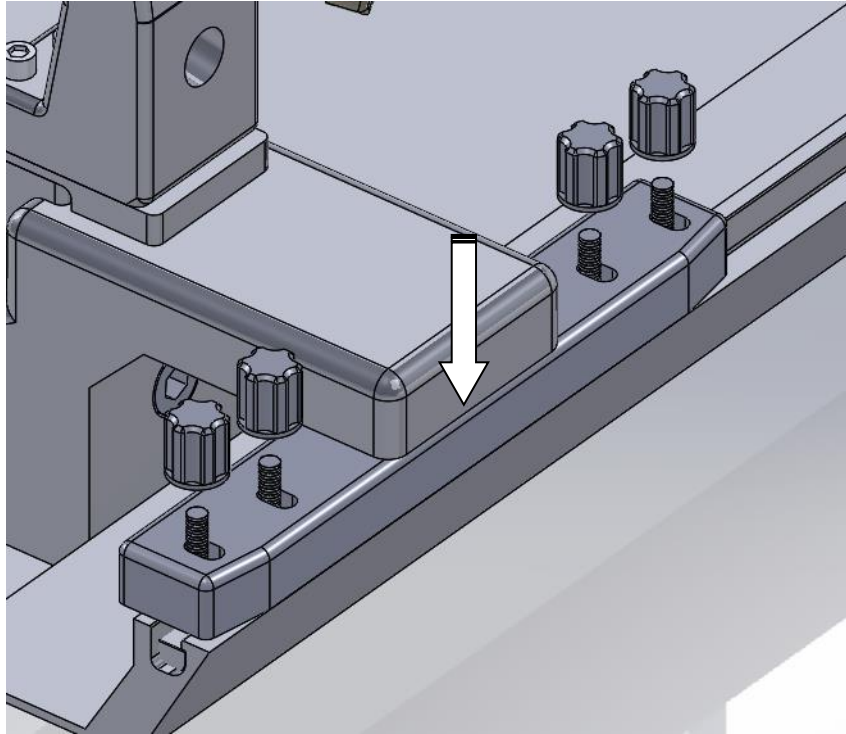
3. Once all 8ea SMS Sola Plug Screws are aligned with the 2ea Table Mounts – Lower the Head Frame down onto the Cradle.

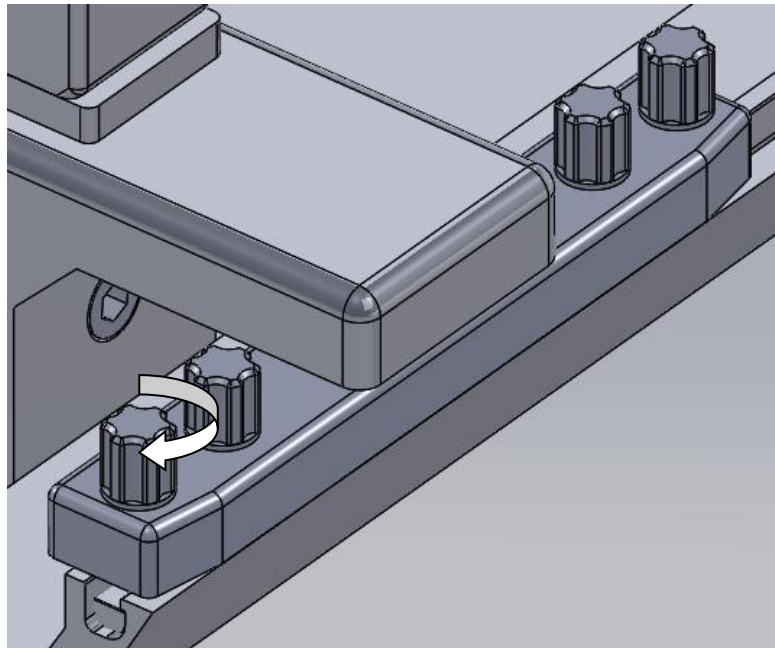4. Fasten the Head Frame to the Cradle using the 8ea Knobs.

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ClearPoint Neuro, Inc.