CLEARPOINT®
NEURO

Flexible Adhesive Equipment Cover, 16x24

INSTRUCTIONS FOR USE

Table of Contents

- 1. Intended Use**
- 2. Device Description**
- 3. General Warnings and Precautions**
- 4. Use Instructions**
 - 4.1. Pre procedure**
 - 4.2. Procedure - Use**
 - 4.3. Post procedure**

1. **Intended Use**
The CLEARPOINT® Flexible Adhesive Equipment Cover is intended to protect equipment from contamination during procedures.

2. **Device Description**

NGS-EC-16X24-06
20 Each

Package Contents

Flexible Adhesive Equipment Cover, 16x24, 3 mil
NON-STERILE

3. **General Warnings and Precautions**

Warning: Do not use any ferrous tools to trim or punch holes in the Equipment Cover while in the scanner room to prevent the risk of the tools being attracted to the magnetic source of the scanner and possibly injuring people or damaging equipment.

MRI Environment Compatibility:

The Equipment Cover is considered MR Safe and can be scanned safely under a static magnetic field.

NOTE: Any serious incident that has occurred in relation to the Equipment Cover should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established

4. **Use Instructions**

NOTE: The following instructions specifically address the method for covering Magnetic Resonance Imaging (MRI) Flexible Coils. Similar methods can be used to cover other instruments as necessary. The Adhesive Equipment Cover is considered single use.

- 4.1. **Pre procedure**

CAUTION: Only non-magnetic materials may be used near the scanner.

4.1.1. When scissors or other cutting instruments are used in preparation of the Equipment Cover, do not perform in the scanner room.

4.1.2. Clear an area approximately 2 ft x 2 ft.

4.1.3. Un-roll one sheet (16" x 24") and lay it down with the adhesive side facing upwards.

Note: A label is applied on the 4th to last sheet to serve as a reminder that the Equipment Cover has been nearly consumed.

Locating on Large Flex Coils

4.1.4. Center the flex coil over the Equipment Cover and press down gently on the coil and around the connector / cable area.

4.1.5. Place another piece of Equipment Cover sticky side down and press gently down onto the coil.

Locating on Small Flex Coils

4.1.6. Center the flex coil over one side of the Equipment Cover and press down gently on the coil and around the connector / cable area.

4.1.7. Fold the other side over on top of the coil, matching the perimeter, and press gently down onto the coil.

4.1.8. Perimeter attachment procedure.

4.1.8.1. Press the excess Equipment Cover around the perimeter firmly together.

4.1.8.2. Press carefully around the exiting cable to ensure a good seal.

4.1.8.3. Trim off an excess that is not desirable with scissors.

4.1.9. Coil gap locations procedure.

4.1.9.1. Gaps that will be pierced to allow fixation screws should have both sheets firmly pressed together.

4.1.9.2. Gaps that will not be breached should have only light pressure applied. The adhesion of the Wrap to the coil should be sufficient to hold it in place without adhering the Wrap to itself at every gap.

- 4.2. **Procedure use.**

4.2.1. Use the flex coil as intended.

4.2.2. Create any openings that are necessary.

4.2.3. Re-seal any separation that might occur around the perimeter of the coil or in the gaps in the coil when noted.

- 4.3. **Post Procedure.**

NOTE: Be careful when cutting around the coil to prevent cutting into the coil or any cabling coming out of the coil.

NOTE: Secure any cabling when removing Equipment Cover in that area to prevent damage to the cable connection.

4.3.1. Remove the flex coil from the scanner room

4.3.2. Trim around the exterior to release the Equipment Cover where it has adhered to itself.

4.3.3. Gently pull the remainder of the Equipment Cover off the coil.

4.3.4. Dispose of the Equipment Cover with biological waste per institution procedure.

4.3.5. Place the tube with remaining Equipment Covers back into the plastic bag / sleeve. Tuck the sleeve into the end of the tube to keep the Equipment Covers clean. The tube can be stored in this manner or in the carton it came in.

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ClearPoint Neuro, Inc.